DE LARYVOX® SECUTRACH® DUSCHESCHUTZ

I. VORWORT

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EN LARYVOX® SECUTRACH® SHOWER GUARD

I. FOREWORD

These instructions are valid for the LARYVOX® SECUTRACH® shower guard. These instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technicaly correct handling. **Please read the instructions for use carefully before using the product for the first time!**

II. INTENDED USE

The LARYVOX® SECUTRACH® shower guard consists of a plastic housing and serves to protect the tracheostoma while showering and washing hair. The shower guard prevents water from entering the tracheostoma and is designed for use with a 22 mm connector ring.

III. WARNINGS

The LARYVOX® SECUTRACH® shower guard is a single-patient product and is thus intended exclusively for one-time or repeated use in one individual patient. Heating to over 65° C, disinfection with boiling water or steam sterilisation are not allowed and lead to damage of the shower guard.

IV. CAUTION
If the LARYVOX® SECUTRACH® shower guard shows damages, cracks or fissures, the product must no longer be used and must be replaced with a new one. Only use the LARYVOX® SECUTRACH® shower guard with the respective recommended aids, such as for instance LARYVOX® Tape Flexible base plates.

V. CONTRAINDICATION

The LARYVOX® SECUTRACH® shower guard serves exclusively to protect the tracheostoma while showering and washing hair. It offers no protection while bathing if the tracheostoma is accidentally immersed under water.

CAUTION! The LARYVOX® SECUTRACH® shower guard must never be used while taking a full bath or during water sports activities.

VI. COMPLICATIONS

Entry of even small amounts of water into the tracheostoma causes an urge to cough due to the lungs' natural protective reflexes. If this is the case, the user must leave the shower immediately and stop showering or washing hair. The LARYVOX® SECUTRACH® shower guard must then be repositioned.

VII. INSTRUCTIONS

Check whether the packaging and the product are in a faultless condition. Check the use-by or expiry date. Do not use the product after this date.

Check whether the shower guard is compatible with the base plate used and remove the filter medium/speaking valve.

Fit the LARYVOX® SECUTRACH® shower guard perpendicularly onto the 22-mm connector of the base plate, exerting gentle pressure.

Always maintain an upright posture while showering. Remove the LARYVOX® SECUTRACH® shower guard after showering. To do this, fix the base plate in position with one hand, hold the shower guard by the gripping recesses and then remove it by pulling it upwards by the trigger.

Now reinsert your filter medium/speaking valve into the base plate as described in the relevant instructions for use.

VIII. CLEANING
The LARYVOX® SECUTRACH® shower guard must be cleaned thoroughly after each use in order to prevent functional impairment due to adhering secretion residues and to minimise the risk of infection. A mild, pH-neutral washing lotion can be used for cleaning. After cleaning, rinse the LARYVOX® SECUTRACH® shower guard thoroughly several times with clear water and then dry it with a clean, soft, lint-free cloth. Do not use any sharp-edged tools for cleaning, store the dried LARYVOX® SECUTRACH® shower guard in a clean, resealable bag.

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