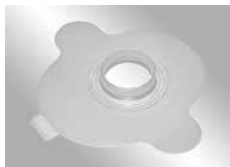
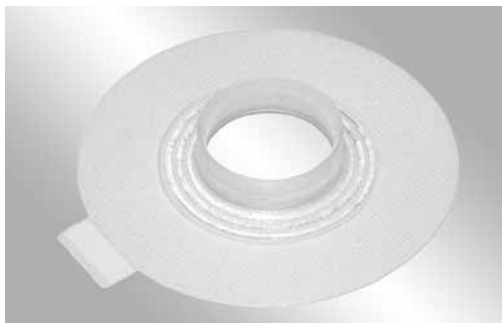
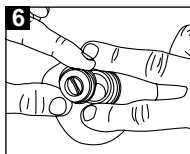
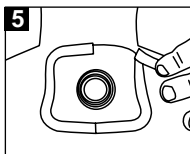
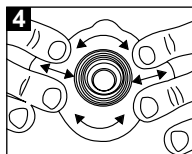
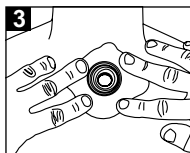
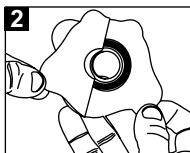
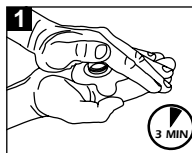
LARYVOX® TAPE

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EN

I. Foreword

These instructions for use are valid for LARYVOX® Tape. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

II. Intended Use

LARYVOX® Tape is an attachment system for daily use of HME filter cassettes or tracheostoma valves with a 22 mm adapter.

III. Warnings

LARYVOX® Tape is a single-patient product and is intended for single use only.

IV. Complications

The circular connector opening can under certain circumstances reduce the usable diameter of the tracheostoma, resulting in a reduction of air supply. The particular properties of the adhesive used for the medical tape can in individual cases cause skin irritations.

V. Precautions

Observe the respective manufacturer's instructions when using tracheostoma valves.

VI. Contraindications

Do not use LARYVOX® Tape Hydrosoft when showering, even if combined with a shower guard. The water solubility of the material will result in unintended detachment of the tape and possible entry of water into the tracheostoma. Use of the tape should be avoided while undergoing radiotherapy due to the risk of skin irritations. After undergoing radiotherapy, only use LARYVOX® Tape after consulting your treating physician. Start using the tape at the earliest 2 weeks after completing radiotherapy.

VII. Product description

LARYVOX® Tape is an attachment system for daily use of HME filter cassettes or tracheostoma valves. The self-adhesive tapes are selected according to the anatomical situation of the tracheostoma and the type of skin in order to achieve a long wearing period and optimal attachment. The filter cassette is fitted in place in the 22 mm combined adapter and can be changed separately from the base plate. We offer the tapes in various shapes and sizes, made of different materials and with different adhesive strengths.

Properties of LARYVOX® Tape STANDARD:

- Shape: round, oval, XL oval
- Good adhesive strength
- Transparent appearance
- Water-repellent effect
- Suitable in situations with unproblematic tracheostoma anatomy

Properties of LARYVOX® Tape FLEXIBLE:

- Shape: round, oval, XL oval
- High adhesive strength
- Suitable for problematic tracheostoma situations

Properties of LARYVOX® Tape HYDROSOFT:

- Shape: round, oval, XL oval
- Particularly skin-friendly
- High fixation comfort
- Ability to enclose fluids/secretions
- Not suitable for use in combination with shower guard

Properties of LARYVOX® Tape COMFORT:

- Shape: round, oval, XL oval
- Highest possible adhesive properties
- Adapts comfortably to movements
- High flexibility and long wearing period
- Optimal for physical activity

Properties of LARYVOX® Tape HYPOALLERGEN:

- Shape: round, oval, XL oval
- Particularly for sensitive skin
- Reduces allergic skin reactions
- Adapts comfortably to movements
- Very good adhesive properties
- High flexibility and long wearing period

Properties of LARYVOX® Tape EXTRA FINE:

- Shape: rectangular
- Very good adhesive properties
- Transparent and extremely thin base plate
- Water-repellent material
- Covers a large skin area around the tracheostoma
- Adapts optimally to the movements of the neck region
- Discreet appearance

VIII. Instructions

1. Clean the skin around the tracheostoma in front of a mirror using warm water and pH-neutral soap or Fahl stoma cleaning cloths. Dry the skin. Before cleaning, remove residues of adhesive from the skin if necessary, using Opticlear®.
2. To protect the skin from irritation, apply Optigard® to the affected area of the skin.
3. The LARYVOX® base plates should be warmed up for 3 minutes before use by holding them between the palms of the hands in order to optimise the adhesive properties and to facilitate adaptation to the affected area of skin ❶ (does not apply to LARYVOX® Tape Extra Fine).
4. After removing the protective foil from the reverse side, ❷ bring the lower edge of the opening of the LARYVOX® Tape into position level with the lower edge of your tracheostoma ❸ and then fix the tape in place on the skin.

5. To ensure optimal fixation and to avoid air enclosures, carefully stroke the entire tape surface with your fingers for 5 minutes ④ after attaching the tape to the skin.
6. Optimal skin attachment is achieved after 20 minutes. Only start inserting the filter cassette when this time has elapsed. This prevents unwanted air enclosures between the base plate and the skin.
7. **Caution! When using LARYVOX® Tape Extra Fine: only remove the protective foil frame on the front side after the base plate has been completely fixed in place ⑤.**
8. To remove a filter cassette, fix the LARYVOX® Tape in position with 2 fingers ⑥ and carefully take out the filter cassette in order to achieve a long wearing period of the tape.

Caution!

Do not use creams that contain fat! Otherwise, secure attachment of the tape is no longer ensured.

IX. Service Life

Thanks to the skin-friendly material, a wearing period of 24 hours or longer is possible. The tape should be used until it comes off the skin. When skin irritations occur, the LARYVOX® Tape must be changed immediately. Deviations from the expected wearing period occur as a result of individual wearing conditions and skin properties.

X. Legal notices

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for functional deficiencies, injuries, infections, and/or other complications or other adverse events caused by unauthorised product alterations or improper use, care, and/or handling.

If the LARYVOX® Tape neck tapes are used beyond the period of use specified under Section IX, and/or if they are used or maintained (cleaned) in non-compliance with the instructions and specifications laid down in these instructions for use, Andreas Fahl Medizintechnik-Vertrieb GmbH will, as far as legally permissible, be free of any liability, including liability for defects. Sale and delivery of all Andreas Fahl Medizintechnik-Vertrieb GmbH products is carried out exclusively in accordance with our General Terms and Conditions of Business which can be obtained directly from Andreas Fahl Medizintechnik-Vertrieb GmbH.

Product specifications described herein are subject to change without notice. LARYVOX® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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