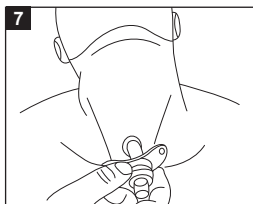
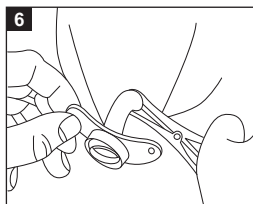
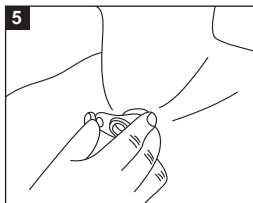
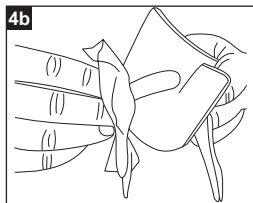
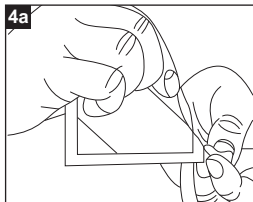
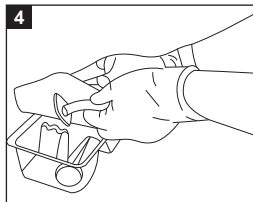
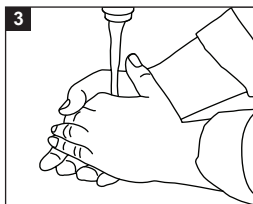
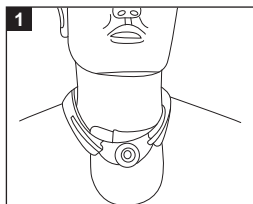
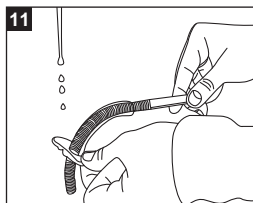
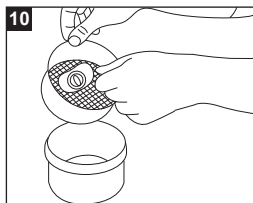
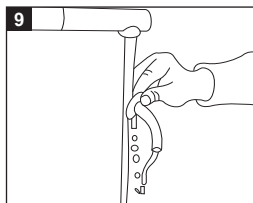
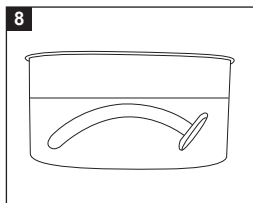
FAHL

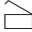
SILVERVENT®

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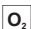
PICTROGRAM LEGEND

EN

	Manufacturer
	Date of manufacture
	Consult instructions for use
	Batch code
	Catalogue number
	Content (in pieces)
	Single patient use
	Keep dry
	Keep away from sunlight
	No inner cannula
	With one inner cannula
	Inner cannula with low profile
	With two inner cannulas
	Inner cannula with 15 mm-connector (UNI)
	Inner cannula fenestrated
	Sieving
	Speaking valve (PHON)
	With oxygen support
	Medical devices

SILVERVENT® TRACHEOSTOMY TUBE

I. FOREWORD

These instructions for use are valid for SILVERVENT® tracheostomy tubes. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling of SILVERVENT® tracheostomy tubes.

Please read the instructions for use carefully before using the product for the first time!

Keep them in an easily accessible place for future reference.

Please keep the package for as long as you use the tracheostomy tube. It contains important information on the product!

II. INTENDED USE

SILVERVENT® tracheostomy tubes are intended to stabilise the tracheostoma following laryngectomy or tracheostomy.

The tracheostomy tube is designed to keep the tracheostoma open.

When used for the first time the products must only be selected, used and inserted by clinicians or medical professionals trained in their use.

Overlong tubes can be indicated particularly in cases of deep-seated tracheal stenoses.

FAHL® LINGO tube variants are intended for tracheotomised patients who retain their larynx or for laryngectomees who use a shunt valve (voice prosthesis users).

III. WARNINGS

Patients must be trained by medical professionals on how to use the SILVERVENT® tracheostomy tubes safely.

SILVERVENT® tracheostomy tubes must never be blocked, e.g. by mucus or encrustation. Blockage can cause asphyxiation!

Mucus in the trachea can be aspirated (removed by suction) through the tracheostomy tube with a tracheal suction catheter.

SILVERVENT® tracheostomy tubes are reusable medical products intended for single-patient use.

Do not re-use with another patient. Do not reprocess for re-use by another patient.

Damaged tracheostomy tubes must not be used and must be discarded immediately. Use of a damaged tube may result in airway compromise.

When the tracheostomy tubes are inserted or removed, this can lead to irritations, coughing or slight bleeding. If bleeding persists, consult your doctor immediately!

Do not use tracheostomy tubes during laser treatment (laser therapy) or with electro-surgical devices! Contact with a laser beam may damage the tube.

In the presence of oxygen-rich mixtures the tube may ignite, which may cause injury from heat and exposure to decomposing and toxic combustion products.

CAUTION!

Tracheostomy tubes with speech function are only recommended for tracheotomised patients with normal secretion and normal mucous tissue.

CAUTION!

If there is strong secretion, a tendency to form granulation tissue, if the patient is undergoing radiation therapy or if scabs have formed, a perforated cannula version is only recommended under regular surveillance by a doctor and provided that shorter replacement intervals are observed (as a rule once a week), because the perforations in the outer tube can increase the formation of granulation tissue.

CAUTION!

Modifications to the tracheostomy tube of any kind, in particular perforations as well as repairs to the tube, may only be carried out by the manufacturer himself or by companies expressly authorised for this in writing by the manufacturer! Inexpertly performed work on tracheostomy tubes can lead to serious injuries.

IV. COMPLICATIONS

The following complications could emerge when using this device:

Contamination of the stoma may make it necessary to remove the tube. Contaminations can furthermore cause infections that necessitate the use of antibiotics.

Unintentional inhalation of a tube that was not sized correctly may need to be removed by a doctor/physician. If mucus blocks the tube it should be removed and cleaned.

V. CONTRAINDICATIONS

Do not use if the patient is allergic to the material.

CAUTION!

Tracheostomy tubes with speaking valves must not be used on any account by laryngectomised patients, as this could lead to serious complications, even suffocation!

SILVERVENT® tracheostomy tubes are not intended for mechanical ventilation!

VI. PRECAUTIONS

The correct tube size should be selected by the doctor/physician or the trained medical professionals.

Do not perform any type of adjustment, repair or alteration on a SILVERVENT® tracheostomy tube or compatible accessories. If a product is dam-aged, it must be discarded immediately.

We highly recommend keeping at least one replacement tube on hand to ensure a continuous supply.

VII. PRODUCT DESCRIPTION

SILVERVENT® tracheostomy tubes are made of sterling silver.

We supply the SILVERVENT® tracheostomy tubes in various sizes and lengths.

Please see the size table in the appendix.

SILVERVENT® tracheostomy tubes are reusable medical products intended for single-patient use.

It may be advisable to use tubes of different lengths alternately so the tip of the tube does not always touch the same spot in the trachea and cause irritation. This prevents pressure points or the formation of granulation tissue. Always discuss the exact procedure with your doctor/physician.

NOTE ON IRRADIATION / RADIOTHERAPY

Tracheostomy tubes made of sterling silver must under no circumstances be used during irradiation (radiotherapy, X-ray, MRI=Magnetic Resonance Imaging), as this can cause severe skin damage!

1. Neck flange

A particular feature of the SILVERVENT® tracheostomy tube is the specially shaped neck flange, which is designed to fit the anatomy of the neck.

The size specifications are printed on the neck flange.

The neck flange of the tracheostomy tubes has two side eyelets for inserting a tube holder.

All SILVERVENT® tracheostomy tubes with fastening eyelets include a disposable tube holder for single use. The tube holder secures the tracheostomy tube to the neck.

Please follow the instructions provided with the tube holder carefully when attaching and removing it from your tracheostomy tube.

Ensure that the SILVERVENT® tracheostomy tube is placed in the tracheostoma without tension and the position is not changed when fastening the tube holder.

2. Connectors and adapters

Connectors and adapters are used to connect compatible tube accessories.

The options available to a patient depend on the clinical condition, such as status post laryngectomy or tracheostomy.

Connectors/adapters are normally permanently attached to the inner cannula. The connector in this case is the universal (15 mm) connector with the aid of which so-called artificial noses or heat and moisture exchangers (HMEs, filters for exchanging heat and moisture) can be attached.

CAUTION!

SILVERVENT® tracheostomy tubes may only be connected to compatible accessories.

3. Cannula tube

The cannula tube borders directly on the neck flange and directs the airflow into the trachea.

The tube tip is rounded to prevent irritation of the mucus membranes in the trachea.

4. Inner cannula

Depending on their specification, the inner cannulas are either equipped with a standard attachment or permanently connected to specific adapters/connectors, or can be connected to detachable accessories such as for instance compatible HMEs.

The inner cannulas can easily be removed from the outer cannula, thus allowing the air supply to be increased quickly if necessary (for instance in case of dyspnoea).

In the case of the silver tracheostomy tubes, the inner cannulas are held in place in the outer cannula by means of a lever. To remove the inner cannula, this lever must be turned sideways.

Inner cannulas must never be used without outer cannula but must always be affixed to the outer cannula.

4.1 Speaking valves

SILVERVENT® tracheostomy tubes as speaking cannulas (LINGO/PHON) with speaking valve are used after tracheotomies with a complete or partially retained larynx and enable the user to speak.

In tracheostomy tubes with silver speaking valve, the speaking valve can be detached from the inner cannula by pushing it out.

VIII. DIRECTIONS FOR TUBE INSERTION AND REMOVAL

For the Doctor/Physician

The appropriate tracheostomy tube must be selected by a doctor/physician or trained medical professionals.

Select a tube that fits the patient's anatomy to optimise comfort and ventilation (breathing in and out).

The inner cannula can be removed at any time to increase air supply or for cleaning. This can for instance be necessary if the cannula is clogged up with secretion residues which cannot be removed by coughing or because no equipment for suctioning off the secretions is available.

For the Patient

Carefully examine the tube before first use to make sure that it is not damaged and that there are no loose parts.

Should you notice any anomaly or anything unusual, DO not use the tube. Return the tube to the manufacturer for inspection.

Tracheostomy tubes must be cleaned thoroughly every time before being inserted. Cleaning is also recommended prior to first use if the tracheostomy tube is not supplied as a sterile product!

The tube must always be cleaned and, if necessary, disinfected as follows before re-inserting according to the instructions provided below.

If secretion collects in the lumen of the SILVERVENT® tracheostomy tube or stoma button and cannot be removed by coughing or aspiration, the tube should be removed and cleaned.

After cleaning and/or disinfection, carefully examine the SILVERVENT® tracheostomy tube for sharp edges, cracks, or other signs of damage, since these may impair function and/or injure the mucus membranes in the airways.

Never under any circumstances continue using damaged tracheostomy tubes.

1. Insertion of the tube

Step-by-step instructions to insert SILVERVENT® tracheostomy tubes.

Before application the users should clean their hands (see picture 3).

Remove tube from the package (see picture 4).

Next, push a tracheal compress onto the tracheostomy tube.

To increase the lubricity of the tracheostomy tube and thus facilitate insertion into the trachea, it is recommended to wipe the outer tube with an OPTIFLUID® stoma oil wipe (REF 31550), which allows even distribution of the stoma oil on the tube (see Figs. 4a and 4b) or FAHL® OPTIFLUID® Lubricant Gel 20 g tube (REF 36100) or FAHL® OPTIFLUID® Lubricant Gel 3 g sachet (REF 36105).

If you are inserting the tube yourself, use a mirror to make insertion of the SILVERVENT® tracheostomy tube easier.

When inserting the SILVERVENT® tracheostomy tube, hold it by the neck flange with one hand (see picture 5).

Pull the tracheostoma slightly apart with your free hand to allow the tip of the tube to fit into the tracheostoma more easily.

Special aids for dilating the tracheostoma are also available (tracheal dilator). These allow the tracheostoma to be dilated gently and evenly, for instance also in emergency situations with collapsing tracheostoma (see picture 6).

Make sure that the tube is not damaged by friction when using an instrument for assistance. Now carefully insert the tube into the tracheostoma during the inspiration phase (while breathing in) and tilt your head slightly back while doing this (see picture 7).

Advance the tube into the trachea.

Straighten your head once the tube has been inserted further into the trachea.

The tracheostomy tubes should always be attached with a special tube holder. This stabilises the tube and thus ensures that the tracheostomy tube is securely seated in the tracheostoma (see picture 1).

2. Removing the tube

CAUTION!

Artificial noses (heat moisture exchanger/HME) must be removed first before proceeding to remove the SILVERVENT® tracheostomy tubes.

CAUTION!

If the tracheostoma is unstable, or in emergency situations (puncture/dilation tracheostomy), the tracheostoma can collapse after withdrawal of the tracheostomy tube, thereby impairing air supply. A fresh tracheostomy tube must be kept ready for use in such cases and must be quickly inserted if necessary. A tracheal dilator can be used for temporarily securing the air supply.

Proceed very carefully to avoid injury to the mucus membranes.

The tube must always be cleaned and, if necessary, disinfected as follows before re-inserting according to the instructions provided below.

Step-by-step instructions to remove the SILVERVENT® tracheostomy tubes:

The tracheostomy tubes should be removed with the head slightly tilted back. To remove the tube, grip the tube at the side by the neck flange / the housing (see picture 7).

Carefully remove the tracheostomy tubes.

In the case of the silver tracheostomy tubes, the inner cannulas are held in place in the outer cannula by means of a lever. To remove the inner cannula, this lever must be turned sideways.

The inner cannula can now be removed from the outer cannula by pulling it gently.

Reinsertion of the inner cannula is performed in reverse sequence as described above.

IX. CLEANING AND DISINFECTION

CAUTION!

For reasons of hygiene and to avoid the risk of infection, you should thoroughly clean the SILVERVENT® tracheostomy tube at least twice a day, more often during heavier secretion production.

In case of an unstable tracheostoma, the airways must always be secured before removing the tracheostomy tube and a replacement tube must be kept ready for insertion. The replacement tube must be inserted immediately, even before cleaning and disinfecting the changed tube.

CAUTION!

The tracheostomy tubes must not be cleaned using a dishwasher, a steam cooker, a microwave oven, a washing machine or similar appliances!

Keep this in mind, your personal cleaning schedule, including additional disinfection if required, must always be determined in consultation with your doctor/physician and according to your individual needs.

Regular disinfection is only required if this is indicated from the medical point of view as ordered by the doctor/physician. The reason for this is that the upper airways are not free of microbes in healthy patients either.

General cleaning may be insufficient to meet the special hygiene requirements to prevent infections for patients with specific problems (e.g. MRSA, ORSA etc.), who are subject to an increased danger of re-infection. We recommend disinfection of the tubes according to the instructions provided below. Please consult your physician.

CAUTION!

Cleaning agent and disinfectant residues on the tracheostomy tube can lead to irritations of the mucus membranes or other health impairments.

Tracheostomy tubes are considered as belonging to the category of instruments with hollow cavities. Special care must therefore be taken when performing cleaning or disinfection that the tube is fully wetted with the solution used and is unobstructed (see picture 8).

The solutions used for cleaning and disinfection must be freshly prepared prior to each use.

1. Cleaning

SILVERVENT® tracheostomy tubes must be cleaned / replaced regularly in accordance with the individual patient's needs.

Only use cleaning agents if the tube is outside the tracheostoma.

A mild, pH-neutral washing lotion can be used for cleaning the tube. We recommend using the special cannula-cleaning powder (REF 31100) as directed by the manufacturer.

Never clean the SILVERVENT® tracheostomy tube with cleaning agents that are not approved by the manufacturer. Never use aggressive household cleaners, detergents, high-concentration alcohol or denture cleaners.

This may be very hazardous to health. The tube may also be destroyed or damaged.

The silver tracheostomy tubes should not be placed in physiological saline solution, as discolourations (silver chloride) can form after longer contact.

Cleaning Steps

Any attached aids are to be removed prior to cleaning.

The inner cannula must also be removed from the outer cannula.

The inner cannula is to be cleaned in the same manner as the outer cannula.

First rinse the tube or button under lukewarm running water (see picture 9).

Use only lukewarm water to prepare the cleaning solution and observe the instructions for use of the cleaning agent.

We recommend the use of the cannula-cleaning tub with sieve insert (REF 31200) to facilitate the cleaning procedure.

Hold the sieve insert by the top edge to prevent contact with and contamination of the cleaning solution (see picture 10).

Place only one tube or button at a time in the sieve insert of the cannula-cleaning tub. If more than one tube is cleaned at a time, they may be compressed too strongly and damaged by excessive pressure.

You can place the inner and outer cannula side by side for this procedure.

Immerse the sieve insert with the tracheostomy tube components in the prepared cleaning solution.

After the immersion time has expired (see instructions for use of the cannula cleaning powder), wash the tracheostomy tube thoroughly several times with lukewarm, clear water (see picture 9). There must be no residues of the cleaning agent on the tube when it is inserted into the tracheostoma.

If necessary, for example, if persistent secretion residues cannot be removed by the cleaning bath, additional cleaning with a special cannula-cleaning brush (OPTIBRUSH®, REF 31850 or OPTIBRUSH® PLUS with fibre top, REF 31855) may help. Only use the cleaning brush, if the tube is removed and already outside the tracheostoma.

Always insert the brush into the tube from the tube top (see picture 11).

Use this brush as directed and proceed with great care to avoid damage to the sensitive tube material.

In tracheostomy tubes with speaking valve, the speaking valve must first be detached.

The valve itself must not be cleaned with the brush since it could otherwise be damaged or break off.

Thoroughly rinse the tracheostomy tube under lukewarm running water.

Dry the tube thoroughly with a clean and lint-free cloth after wet cleaning.

Never use tracheostomy tubes with impaired functionality or with damages such as sharp edges or cracks, because this may lead to injuries of the mucus membranes in the trachea. If damages are found, the tube must not be used under any circumstances.

2. Disinfection Instructions

Disinfection should always be done if ordered by a doctor/physician due to specific health concerns caused by disease, infection, or your specific situation.

Disinfecting is generally indicated to prevent cross-infection and in inpatient situations (e.g. hospitals, nursing homes, and/or other health care facilities) to limit infections.

CAUTION!

Always clean according to the cleaning procedure described above before proceeding to disinfection (if applicable).

Disinfectants that release chlorine or that contain strong alkalis or phenol derivatives must under no circumstances be used. This could badly damage or even destroy the tracheostomy tube.

Disinfection steps

For disinfection, place the silver cannula in boiling water (100 degrees Celsius or 212 degrees Fahrenheit) for 5 to 10 minutes. Make sure that this completely covers the silver cannula.

Dry the tube thoroughly with a clean and lint-free cloth after wet cleaning.

CAUTION!

Allow the tube to cool down before inserting it again to avoid scalds and burns.

X. STORAGE/CARE

Currently not used and cleaned tubes should be stored in a clean plastic container, dry environment away from dust, sunlight and heat.

After the inner cannula has been cleaned, disinfected if necessary and then dried, the outer surface of the inner cannula should be lubricated by applying stoma oil, e.g. using an OPTIFLUID® stoma oil wipe.

Only use stoma oil (OPTIFLUID® Stoma Oil, 25 ml bottle REF 31525 / Stoma oil wipe REF 31550) or lubricant gel (FAHL® OPTIFLUID® Lubricant Gel 20 g tube REF 36100/ 3 g sachet REF 36105) as lubricant.

XI. SERVICE LIFE

SILVERVENT® tracheostomy tubes made of sterling silver are designed for long-term use. Send us your tracheostomy tubes for checking and reconditioning on a regular basis, at least every 6 months. The service life depends on the intensity of use as well as on maintenance and regular checking and reconditioning. However, the tracheostomy tubes should be replaced at the latest 3 years after first use.

The service life of a tube is influenced by many factors. The composition of the secretions, the quality of cleaning and other aspects, for example, are very important.

The maximum service life is not extended by using the tracheostomy tube intermittently (e.g. by alternating with other tubes at normal cleaning intervals).

Damaged tubes must be replaced immediately.

XII. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for functional deficiencies, injuries, infections, and/or other complications or adverse events caused by unauthorised product alterations or improper use, care, and/or handling.

In particular, Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by modifications to the tracheostomy tube, especially due to shortening of the same or perforations, if these modifications or repairs were not carried out by the manufacturer himself. This applies both to damages to the tubes themselves caused thereby and to any consequential damages caused thereby.

If the tracheostomy tube is used beyond the period of use specified under Section XI, and/or if the tracheostomy tube is used, maintained (cleaned, disinfected) or stored in non-compliance with the instructions and specifications laid down in these instructions for use, Andreas Fahl Medizintechnik-Vertrieb GmbH will, as far as legally permissible, be free of any liability, including liability for defects.

Should a serious event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

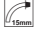
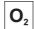
Sale and delivery of all Andreas Fahl Medizintechnik-Vertrieb GmbH products is carried out exclusively in accordance with our General Terms and Conditions of Business which can be obtained directly from Andreas Fahl Medizintechnik-Vertrieb GmbH.

Product specifications described herein are subject to change without notice.

SILVERVENT® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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All measurements listed in the tables below are for standard cannulas.
Customised cannulas are available upon request.

GR	AD1	AD2	ID1	ID2	L1	L2	θ-BW
00	6,1	4,5	3,7	2,6	50,0	45,0	90°
01	6,4	5,0	4,2	3,1	50,0	45,0	90°
02	6,8	5,7	4,9	3,8	50,0	45,0	90°
03	7,5	6,4	5,6	4,6	55,0	49,0	90°
04	8,2	7,0	6,2	5,2	55,0	49,0	90°
05	9,1	7,7	6,9	5,9	60,0	53,0	90°
06	9,8	8,4	7,6	6,5	60,0	53,0	90°
07	10,9	9,0	8,2	7,1	65,0	58,0	90°
08	11,6	9,7	8,9	7,7	70,0	60,0	90°
09	12,3	10,4	9,6	8,7	75,0	65,0	90°
10	13,2	11,0	10,2	9,3	80,0	68,0	90°
11	13,8	11,7	10,9	10,0	90,0	77,0	90°
12	14,6	12,4	11,6	10,6	90,0	77,0	90°
13	15,0	13,0	12,2	11,3	90,0	77,0	90°

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Sizing information:

All lengths in mm | GR = Size | AD1 = Outer diameter at the neck flange | AD2 = Outer diameter at cannula tip | ID1 = Inner diameter at the cannula tip of the outer cannula | ID2 = Inner diameter at the cannula tip of the inner cannula | L1 = Length over the outer curve | L2 = Length over the middle curve | θ-BW = Bending angle

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