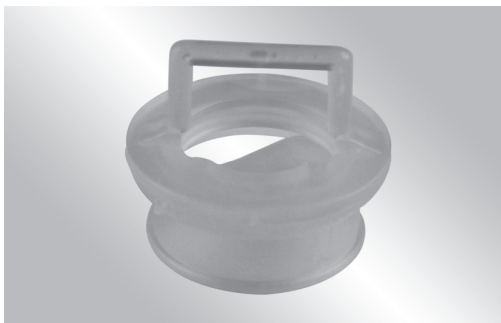
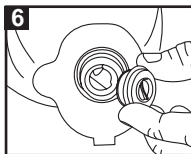
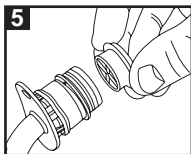
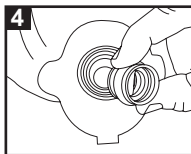
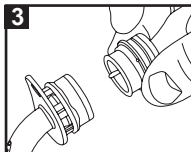
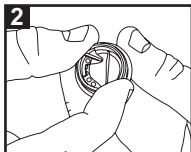
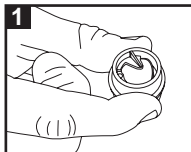
## LARYVOX® HANDS-FREE VALVE

---

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EN

## I. Foreword

These instructions for use are valid for **LARYVOX® Hands-Free** speaking valves. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

## II. Intended Use

The adjustable **LARYVOX® Hands-Free Valve** enables patients with partially or fully preserved larynx as well as persons wearing voice prostheses (shunt valve) to speak without having to use their fingers. The **LARYVOX® Hands-Free Valve** consists of a plastic housing, a silicone diaphragm disc and a flexible housing ring, with which the closing sensitivity of the silicone diaphragm can be adjusted. The valve is intended for use exclusively in combination with a phonation cannula with 22 mm adapter as well as with LARYVOX® Tape base plates.

When used for the first time, the products must be selected, employed and inserted by physicians or medical professionals trained in their use, and the user must be instructed in this regard by such medical professionals.

## III. Warnings

- Speaking valves are single-patient products and are thus intended exclusively for use by **one** patient only. They can be used several times by the same patient.
- Re-use of any kind in **other** patients and thus also reconditioning for re-use in **other** patients is not allowed.
- Clean the **LARYVOX® Hands-Free** speaking valves only with cleaning agents approved by the manufacturer.
- The **LARYVOX® Hands-Free Valve** cannot be sterilised.
- Heating to over 65°C, disinfection with boiling water or steam sterilisation are not allowed and lead to damage of the speaking valve.

## VI. Complications

- If hands-free speaking is not possible for you despite the largest possible diaphragm setting, or if you find hands-free speaking difficult logopaedic training may be necessary. Please consult your physician.
- If the pressure is too high, e.g. if you are coughing, the valve diaphragm (silicone diaphragm disc) may be forced through the opening in the front plate (housing ring), accompanied by the sound of rapidly escaping air. If this happens, pull the entire speaking valve away from the cannula/base plate and with your fingertips return the valve diaphragm carefully to the original position. Especially if you need to cough, you should always remove the speaking valve to prevent the diaphragm from sticking and thus impairing the function.

- To ensure vocal success, the tracheostoma must be completely sealed. This can be achieved by providing a tracheostomy tube, by a base plate or by a combination of the two (e.g. LARYNGOTEC®, LARYNGOTEC® KOMBI CLIP LINGO and LARYVOX® Tape).
- If the particle filter is clogged or if signs of clogging appear, the **LARYVOX® Hands-Free** speaking valve must immediately be removed and cleaned.

## V. Precautions

Care must be taken to ensure that the valve opening is not blocked by clothing or the like in order to enable unhindered breathing (risk of respiratory distress).

## VI. Contraindication

The **LARYVOX® Hands-Free Valve** is permitted for use exclusively by patients with partially or fully preserved larynx as well as persons wearing voice prostheses (shunt valve)!

The **LARYVOX® Hands-Free Valve** must be removed before going to sleep to prevent potential obstruction of the airways!

## VII. Product description

The **LARYVOX® Hands-Free** speaking valve is available in a number of different versions:

### **LARYVOX® Hands-Free Valve Low Resistance (REF 21984)**

- Only low blowing pressure is required to close the speaking valve
- The strap running across the housing can be used for manual fine tuning of the diaphragm setting and as a spacer for textiles

### **LARYVOX® Hands-Free Valve Normal / Low Resistance for TrachiNaze® (REF 21981/21980)**

- Can be used in combination with TrachiNaze® Plus filters
- The NORMAL RESISTANCE version has a blue valve diaphragm, which is somewhat thicker and heavier than the uncoloured valve diaphragm in the LOW RESISTANCE version
- In the LOW RESISTANCE version, only low blowing pressure is required to close the speaking valve.

### **LARYVOX® Hands-Free Valve Kombi HME Normal / Low Resistance (REF 21983/21982)**

- The integrated combination adapter is used to accommodate all filter cassettes with a 22 mm port, such as LARYVOX® HME filter cassettes or HUMIDOTWIN®, for example.
- In the LOW RESISTANCE version, only low blowing pressure is required to close the speaking valve.

## VIII. Instructions

- Take the valve out of the packaging, clean and dry it before first use in accordance with the instructions (Cleaning and Disinfection) prior to first use.

- The user must check the **Laryvox® Hands-Free Valve** for completeness, absence of damage and proper functioning of the product each time before inserting it again ❶!
- To prevent injury, it is advisable to first connect the speaking valve to the tracheostomy tube and then to insert the tracheostomy tube with attached speaking valve into the tracheostoma. Otherwise, excessive pressure could be exerted on the mucous membranes via the cannula tube when the speaking valve is snapped into place.
- You should adjust the speaking valve before attaching it and set the opening thickness of the valve disc by turning it ❷. The further the valve is opened, the more air can be inhaled. However, the exhalation pressure necessary to close the valve while speaking is then also greater. For normal respiration and normal physical activity, it is therefore recommended that you set the valve such that sufficient air can flow in, but without opening the valve completely. It is recommended that you open the valve to the maximum only for strenuous physical and sporting activities, in which your breathing becomes deeper.
- After you have adjusted the valve disc to the desired opening thickness, insert the speaking valve together with the insertion ring, which can be seen on the small circumferential retaining bead, into the 22 mm adapter of the cannula ❸.
- If special base plates are used for adhesion (e.g. LARYVOX® Tape), first fasten the base plate according to the instructions.
- Then carefully insert the **LARYVOX® Hands-Free Valve** into the retaining ring of the ❹ base plate, so that a secure connection is established.
- If you are using a **LARYVOX® Hands-Free Valve** for TrachiNaze® and **LARYVOX® Hands-Free Valve** Combi HME, now insert the additional filter ❺ / ❻.
- The retaining bead ensures that the speaking valve will be seated stably.
- The strap (**LARYVOX® Hands-Free Valve** Low Resistance version) serves as a spacer to prevent restriction of breathing by clothing or the like. It is not suitable as a handle for positioning the speaking valve.

## IX. Service Life

- The service life of the **LARYVOX® Hands-Free Valve** is influenced by many factors. Thus, the composition of secretions, conscientious cleaning and other aspects can be of decisive significance.
- The **LARYVOX® Hands-Free Valve** has an expected life of twelve month when carefully treated. After that, the product must be replaced.
- A speaking valve showing damage of any kind, even if only minor, must be replaced immediately.
- Speaking valves that are presently not being used should be stored in a dry environment in a clean plastic container, protected from dust, sunlight and/or heat.

## X. Cleaning and Disinfection

### 1. Cleaning

The **LARYVOX® Hands-Free Valve** must be cleaned thoroughly at least twice a day, and in situations with strong secretion production correspondingly more often, to prevent functional impairment of the speaking valve by adhering residues of secretions and to minimise the risk of infection.

#### Cleaning Steps

- Carefully remove the speaking valve by twisting and at the same time pulling it out of the adapter of the tracheostomy tube or from the retaining ring of the base plate.
- Then remove any filter systems that have been used.
- A mild, pH-neutral washing lotion can be used for cleaning the speaking valve. We recommend using the special cannula-cleaning powder OPTIBRUSH® CLEAN (REF 31110) as directed by the manufacturer. We recommend the use of the cannula-cleaning tub OPTIBRUSH® CONT (REF 31200) to facilitate the cleaning procedure.
- After cleaning, rinse the speaking valve thoroughly several times with clear water and then dry it with a clean, soft, lint-free cloth. Do not use any sharp-edged cleaning utensils!
- Alternatively, cleaning by thermal disinfection at not more than 65 °C is also possible. Use clean water with a temperature of not more than 65 °C for this purpose. Make sure that the temperature is kept constant (use a thermometer to monitor the temperature) and never under any circumstances expose the tubes to boiling water for disinfection. This could seriously damage the speaking valve.

### 2. Chemical Disinfection Instructions

- Disinfection should always be done if ordered by a physician due to specific health concerns caused by disease, infection, or your specific situation.
- In order to reduce risks of infection, disinfecting is generally indicated to prevent cross-infection and in inpatient situations (e.g. hospitals, nursing homes, and/or other health care facilities) to reduce risks of infection.

#### Disinfection steps

- For disinfection, the speaking valve must be immersed in a disinfectant solution for 10 to 15 minutes following the cleaning procedure described above.
- Only use OPTICIT® tube disinfectant (REF 31180; not available in the USA) for this, and follow the manufacturer's instructions. As an alternative, we recommend a disinfectant based on glutaric dialdehyde as active ingredient (also available in the USA). Always observe the area of application and spectrum of activity specified for the disinfectant by the manufacturer.
- After disinfection, the **LARYVOX® Hands-Free Valve** must be rinsed inside and out very thoroughly with sterile or chilled, boiled water and then dried.

## XI. General Terms and Conditions

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for functional deficiencies, injuries, infections, and/or other complications or adverse events caused by unauthorised product alterations or improper use, care, and/or handling.

In particular, Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by modifications to the speaking valve or by repairs, if these modifications or repairs were not carried out by the manufacturer himself. This applies both to damages to the speaking valves themselves caused thereby and to any consequential damages caused thereby. If the speaking valve is used beyond the period of use specified under Section IX, and/or if the speaking valve is used, maintained (cleaned, disinfected) or stored in non-compliance with the instructions and specifications laid down in these instructions for use, Andreas Fahl Medizintechnik-Vertrieb GmbH will, as far as legally permissible, be free of any liability, including liability for defects.

Sale and delivery of all Andreas Fahl Medizintechnik-Vertrieb GmbH products is carried out exclusively in accordance with our General Terms and Conditions of Business which can be obtained directly from Andreas Fahl Medizintechnik-Vertrieb GmbH.

Product specifications described herein are subject to change without notice. **LARYVOX®** is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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