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I. FOREWORD

These instructions are applicable for OPTICLEAR® adhesive remover. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

II. INTENDED USE

OPTICLEAR® is a special wipe for removing remnants of skin adhesives of all kinds and is suitable in particular around the tracheostoma.

III. WARNINGS

Flammable. Do not bring into contact with open flames.

OPTICLEAR® is intended only for external use. Do not apply on open wounds or in body orifices. Avoid contact with the eyes. Apply only if the skin is intact.

Do not use on small children.

OPTICLEAR® Adhesive Remover is a single-patient product and is intended only for one single use.

Reuse may impair the function and is therefore not permitted. Should you experience skin irritation (e.g. redness, itches), discontinue the product immediately and consult a physician. This may be due to special skin sensitivity or allergy-related incompatibility.

IV. PRODUCT DESCRIPTION

The non-woven wipe, which can be gripped without slipping, is soaked with an alcohol-base cleaning lotion.

Remnants of adhesives are quickly and gently removed from the skin and absorbed. OPTICLEAR® is therefore particularly suitable for cleaning the skin after adhesive plasters/base plates have been removed.

Ingredients: Isopropylalcohol, Isopropylpalmitate, Parfum

V. INSTRUCTIONS

Using gentle pressure, push the cleaning wipe under the edge of the adhesive plaster/base plate, to allow the fluid it contains to penetrate between the adhesive plaster/base plate and the skin ☺.

Repeat the process until the adhesive plaster/base plate has become completely detached harmlessly from the skin.

Thereupon clean the skin with water and dry it carefully.

VI. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use.

Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

Product specifications described herein are subject to change without notice. OPTICLEAR® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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