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#### en PICTOGRAM LEGEND

**Pictograms listed below you'll find on the product packaging, if applicable.**

<span><span><span></span></span></span> Catalogue number
<span><span><span></span></span></span> Batch code
<span><span><span></span></span></span> Use by
<span><span><span></span></span></span> Content (in pieces)
<span><span><span></span></span></span> Consult instructions for use
<span><span><span></span></span></span> Keep away from sunlight
<span><span><span></span></span></span> Keep dry
<span><span><span></span></span></span> For reuse on a single patient
<span><span><span></span></span></span> Medical devices
<span><span><span></span></span></span> CE marking
<span><span><span></span></span></span> Manufacturer

#### 1. FOREWORD

These instructions for use are valid for **OPTIBRUSH®**, **OPTIBRUSH® Basic** & **OPTIBRUSH® PLUS** cannula cleaning brush.

These instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

Please read the instructions for use carefully before using the product for the first time.

Keep them in an easily accessible place for future reference. Please keep the package for as long as you use the product. It contains important information on the product!

##### 2. SAFETY INFORMATION

Always remove the cannula from the tracheostoma before commencing cleaning.

Never insert the cannula cleaning brush directly into the tracheostoma!

Do not repair or modify the brush.

Damaged products need to be properly disposed immediately!

**OPTIBRUSH®**, **OPTIBRUSH® Basic** & **OPTIBRUSH® PLUS** is a single-patient product.

##### 3. INTENDED PURPOSE

The **OPTIBRUSH®**, **OPTIBRUSH® Basic** & **OPTIBRUSH® PLUS** is a special brush for gentle cleaning of tracheal canulas.

##### 4. APPLICATION

The **OPTIBRUSH®**, **OPTIBRUSH® Basic** & **OPTIBRUSH® PLUS** should be preshaped prior to initial use according to the shape of the cannula tube.

In order to avoid damage, it is recommended to clean the tracheal canula with the brush starting at the cannula tip. This serves to simply handling.

##### 5. HYGIENE INSTRUCTIONS

For hygienic reasons, the cannula cleaning brush should be thoroughly rinsed with clear water after each use and subsequently stored in a beaker or glass for drying with the handle pointing down!

##### 6. STORAGE

Store in a dry environment away from sunlight and/or heat.

##### 7. SERVICE LIFE

Depending on frequency and level of use, we recommend replacing the cannula cleaning brush on a regular basis. This is especially important when the bristles are worn, discolored or stuck together.

##### 8. DISPOSAL

The product must only be disposed of in accordance with the national regulations.

##### 9. LEGAL NOTICES

The manufacturer Andreas FahI Medizintechnik-Vertrieb GmbH will not accept any liability for damages (especially not for functional deficiencies, injuries, infections, and/or oth-er complications or adverse events) caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use.

This applies – to the extent permitted by law – both to damages to the products themselves caused thereby and to any consequential damages caused thereby.

Product specifications described herein are subject to change without notice.

**OPTIBRUSH®** is a trademark and brand of Andreas FahI Medizintechnik-Vertrieb GmbH, Cologne registered in Ger-many and the EU member states.

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Výrobce Andreas FahI Medizintechnik-Vertrieb GmbH ne-