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**I. FOREWORD**  
These instructions are applicable for OPTIGARD® skin protection film. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. Please read the instructions for use carefully before using the product for the first time!

**II. INTENDED USE**  
OPTIGARD® is a special wipe for preparatory skin care in the use of adhesive plasters/base plates and skin adhesives.

**III. WARNINGS**  
Flammable. Do not bring into contact with open flames. OPTIGARD® is intended only for external use. Do not apply on open wounds or mucous membranes. Avoid contact with the eyes. Apply only if the skin is intact. Do not use on small children. OPTIGARD® Skin Protection Film is a single-patient product and is intended only for one single use. Reuse may impair the function and is therefore not permitted.

**IV. PRODUCT DESCRIPTION**  
OPTIGARD® protects against skin irritations, which can be caused by adhesive plasters/base plates and special skin adhesives. The skin is covered with a thin protective film, which dries after a few seconds.

**Ingredients:** Aqua (water), pvp/VA Copolymer, isopropylalcohol, peg-40 hydrogenated castor oil, parfum.

**V. INSTRUCTIONS**  
First clean and degrease the affected part of the skin in the normal way. Then moisten the skin area completely with the OPTIGARD® Special Wipe ①.

**I. PREMESSA**  
Le presenti istruzioni riguardano il film dermoprotettivo OPTIGARD®. Make sure that the skin is completely dry before applying the adhesive plaster/base plate or if necessary a skin adhesive ②. The film layer does not have to be removed for each change of plaster. For optimum skin protection, however, we recommend washing off the film layer with a mild cleaning lotion for each change of plaster.

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**IV. DESCRIZIONE DEL PRODOTTO**  
OPTIGARD® protegge la pelle da irritazioni che possono essere causate da fasciature adesive/piastre base o speciali adesivi cutanei. This applies - to the extent permitted by law - both to damages to the products themselves caused thereby and to any consequential damages caused thereby. Product specifications described herein are subject to change without notice.

Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

OPTIGARD® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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**III. WARNINGS**  
Do not bring into contact with open flames. OPTIGARD® is intended only for external use. Do not apply on open wounds or mucous membranes. Avoid contact with the eyes. Apply only if the skin is intact. Do not use on small children. OPTIGARD® Skin Protection Film is a single-patient product and is intended only for one single use. Reuse may impair the function and is therefore not permitted.

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