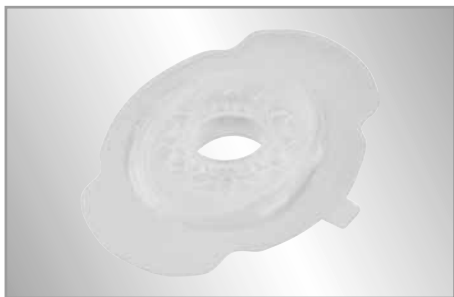
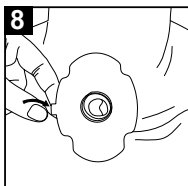
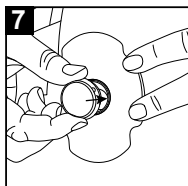
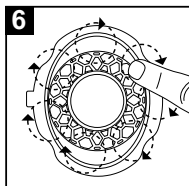
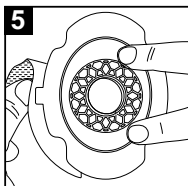
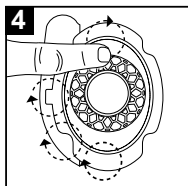
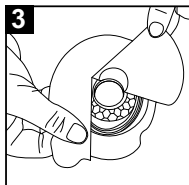
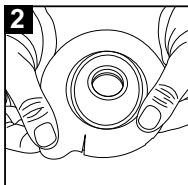
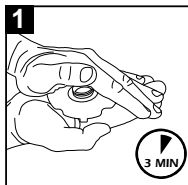
FAHL

LARYVOX® TAPE CONVEX

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LARYVOX® TAPE CONVEX

I. FOREWORD

These instructions for use are valid for LARYVOX® Tape CONVEX. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

II. INTENDED USE

LARYVOX® Tape CONVEX is an attachment system for daily use of HME filter cassettes or tracheostoma valves with a 22 mm adapter.

III. WARNINGS

LARYVOX® Tape CONVEX is a single-patient product and is intended for single use only.

IV. COMPLICATIONS

The circular connector opening can under certain circumstances reduce the usable diameter of the tracheostoma, resulting in a reduction of air supply. The particular properties of the adhesive used for the medical tape can in individual cases cause skin irritations.

V. PRECAUTIONS

Observe the respective manufacturer's instructions when using tracheostoma valves.

VI. CONTRAINDICATIONS

Use of the tape should be avoided while undergoing radiotherapy due to the risk of skin irritations. After undergoing radiotherapy, only use LARYVOX® Tape CONVEX after consulting your treating physician. Start using the tape at the earliest 2 weeks after completing radiotherapy.

VII. PRODUCT DESCRIPTION

LARYVOX® Tape CONVEX is an attachment system for daily use of HME filter cassettes or tracheostoma valves. The self-adhesive tapes are selected according to the anatomical situation of the tracheostoma and the type of skin in order to achieve a long wearing period and optimal attachment. The filter cassette is fitted in place in the 22 mm combined adapter and can be changed separately from the base plate.

Properties of LARYVOX® Tape CONVEX:

- Shape: oval
- Suitable for precise positioning in a deep or deepened tracheostoma
- The base plate as honeycomb core offers a strong overall structure
- Reduces the occurrence of allergic skin reactions
- Transparent appearance
- Water-repellent material
- Very good adhesive properties

VIII. INSTRUCTIONS

1. Clean the skin around the tracheostoma in front of a mirror using warm water and pH-neutral soap or OPTIFAHL® stoma cleaning cloths (REF 33200). Dry the skin. Before cleaning, remove residues of adhesive from the skin if necessary, using OPTICLEAR® (REF 33500).
2. To protect the skin from irritation, apply OPTIGARD® to the affected area of the skin.
3. The LARYVOX® base plates should be warmed up for 3 minutes before use by holding them between the palms of the hands in order to optimise the adhesive properties and to facilitate adaptation to the affected area of skin ❶.
4. Lightly tear the perforation of the protective film approximately 1 cm apart ❷.
5. Remove the large section of the protective film ❸.
6. Position LARYVOX® Tape Convex vertically with the adhesive surface on your neck so that the lower edge of the central opening is aligned at the same height as the lower edge of the tracheostomy ❹. Smooth this area with your finger to ensure that the base plate is positioned correctly.
7. Now remove the other section of the protective film ❺ and smooth the area with your finger ❻.
8. To ensure optimal fixation and to avoid air enclosures, carefully stroke the entire tape surface with your fingers for 5 minutes after attaching the tape to the skin.
9. Optimal skin attachment is achieved after 20 minutes. Only start inserting the filter cassette when this time has elapsed. This prevents unwanted air enclosures between the base plate and the skin.
10. To remove a filter cassette, fix the LARYVOX® Tape in CONVEX position with 2 fingers ❼ and carefully take out the filter cassette in order to achieve a long wearing period of the tape.
11. To remove the LARYVOX® Tape CONVEX, carefully grip an adhesive tab. Carefully peel this off from one side, if necessary with OPTICLEAR® (REF 33500) ❽.

CAUTION!

Do not use creams that contain fat! Otherwise, secure attachment of the tape is no longer ensured.

IX. SERVICE LIFE

Thanks to the skin-friendly material, a wearing period of 24 hours or longer is possible. The tape should be used until it comes off the skin. When skin irritations occur, the LARYVOX® Tape CONVEX must be changed immediately. Deviations from the expected wearing period occur as a result of individual wearing conditions and skin properties.

CAUTION!

Cleaning, disinfection, re-sterilisation and re-use can impair the safety and function of the product and are therefore not permissible!

X. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for functional deficiencies, injuries, infections, and/or other complications or other adverse events caused by unauthorised product alterations or improper use, care, and/or handling. If the LARYVOX® Tape neck tapes are used beyond the period of use specified under Section IX, and/or if they are used or maintained (cleaned) in non-compliance with the instructions and specifications laid down in these instructions for use, Andreas Fahl Medizintechnik-Vertrieb GmbH will, as far as legally permissible, be free of any liability, including liability for defects.

Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled. Sale and delivery of all Andreas Fahl Medizintechnik-Vertrieb GmbH products is carried out exclusively in accordance with our General Terms and Conditions of Business which can be obtained directly from Andreas Fahl Medizintechnik-Vertrieb GmbH. Product specifications described herein are subject to change without notice. LARYVOX® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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