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**EN** These instructions for use are valid for LARYVOX® SIMPLY FIX. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

**II. INTENDED USE** LARYVOX® SIMPLY FIX enables secure fixation of a tracheostomy tube and can be attached variably to the neck area.

**III. WARNINGS** LARYVOX® SIMPLY FIX are single-patient products and are intended for single use only.

**CAUTION!** Cleaning, disinfection, re-sterilisation and re-use can impair the safety and function of the product and are therefore not permissible!

Tracheofix® SIMPLY FIX must only be used in combination with a tracheostomy tube.

To prevent the tracheostomy tube from coming loose by accident, the secure hold of the LARYVOX® SIMPLY FIX must be checked regularly.

Should you experience skin irritation (e.g. redness, itchiness), discontinue the product immediately and consult a physician. This may be due to special skin sensitivity or allergy-related incompatibility.

**IV. INSTRUCTIONS**

1. Clean the skin around the tracheostoma in front of a mirror using warm water and pH-neutral soap or OPTIFAHIL® stoma cleaning cloths (REF: 33200). Dry the skin. Before cleaning, remove residues of adhesive from the skin if necessary, using OPTICLEAR® (REF: 33500).

2. To protect the skin from irritation, apply OPTIGARD® to the affected area of the skin.

3. Attach the two hook-and-loop fasteners to the lateral eyelets on both sides of the neck flange. 4. Now carefully pull the tracheostoma during the inspiration phase (while breathing in) while tilting your head slightly back. 5.

5. Pull off the protective foil on the back of the adhesive surface 6. and attach by applying slight pressure to the foam. 7.

6. If the product is incorrectly/uncomfortably positioned, you can detach and reposition it.

#### CAUTION!

Do not use creams that contain fat! Otherwise, secure attachment of the tape is no longer ensured.

**V. SERVICE LIFE** The maximum period of use is 24 hours. Deviations from the expected wearing period occur as a result of individual wearing conditions and skin properties.

**CAUTION!** The products must be replaced immediately if damage occurs or if a secure hold is no longer guaranteed due to decreasing adhesion.

**VI. LEGAL NOTICES** The manufacturer Andreas FahI Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use.

Should a patient give a suspended vote in connection with this product of Andreas FahI Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

Product specifications described herein are subject to change without notice. LARYVOX® is a German brand name of Andreas FahI Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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© 2019 Andreas FahI Medizintechnik-Vertrieb GmbH. Alle Rechte vorbehalten. LARYVOX® ist eine eingetragene Marke der Andreas FahI Medizintechnik-Vertrieb GmbH, Köln.