## DE I. VORWORT

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## EN I. FOREWORD

These instructions apply to the LARYVOX® NEBULIZER.

The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

**Please read the instructions for use carefully before using the product for the first time!**

### II. INTENDED USE

The LARYVOX® NEBULIZER is a pocket inhaler for generating a spray mist to moisten the mucous membranes of the tracheostoma.

Suitable for use with water, saline solution as well as medications and other inhalants approved for use in inhalation therapy.

### III. WARNINGS

**The LARYVOX® NEBULIZER is a single-patient product and is thus intended exclusively for one-time or repeated use in one individual patient.**

### IV. INSTRUCTIONS

Unscrew the atomiser head of the container.

Fill the inhalant into the container.

### CAUTION!

**Do not fill hot liquids into the container.**

### Danger of scalding!

Screw the atomiser head on the container.

Allow all parts to dry in a clean environment.

For hygienic reasons we recommend daily cleaning.

**Pressing the pump atomiser generates a spray mist with which the mucous membranes of the tracheostoma can be moistened **!****

For better/more precise positioning of the spray mist, the riser tube on the atomiser head can be adjusted **!**.

### CAUTION!

**Do not insert the riser tube into the tracheostoma.**

### Risk of injury!

**CAUTION!** To avoid the formation of bacteria, the inhalant should be changed daily.

### V. CLEANING

Thoroughly rinse the container, the atomiser head and the riser tube under lukewarm, running water.

Allow all parts to dry in a clean environment.

For hygienic reasons we recommend daily cleaning.

### CAUTION!

**Do not use water with a temperature > 60 °C for cleaning, as this can lead to damage to the product.**

### VI. PERIOD OF USE

The service life depends on the frequency of use.

The products must be replaced immediately when damaged.

### VII. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by unauthorised product alterations,

by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use. Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled. Product specifications described herein are subject to change without notice. LARYVOX® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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