

FAHL® OPTIFLUID® Lubricant Gel



STERILE R CE 0482



STERILE R CE 0482

- ▶ Sterile, colorless water-based lubricant
- ▶ The FAHL® OPTIFLUID® Lubricant Gel facilitates ease of insertion of tracheal cannulas into the tracheostomy
- ▶ With help of a clean, lint-free fleece cloth, carefully rub a small amount of lubricant onto the tracheal cannula, which can then be inserted into the tracheostomy as usual
- ▶ Also for use on tracheal cannulas with inner cannulas: rubbing the inner cannula with the lubricant facilitates insertion in the outer cannula
- ▶ The care instructions from the cannula manufacturer should always be observed before using the gel for the first time
- ▶ Apply sparingly: a couple of drops of FAHL® OPTIFLUID® Lubricant Gel should be enough to ease insertion of the outer/inner cannula
- ▶ Available in 20 g tubes (REF 36100) or in practical 3 g sachets (REF 36105) for single use, ideal for travelling and on the go
- ▶ Ingredients: Purified Water, Glycerin, Hydroxyethylcellulose, Phenoxyethanol Ethylhexylglycerin, Imidazolidinyl Urea, Propylene Glycol, Aspartame, Disodium EDTA

Order information:

FAHL® OPTIFLUID® Lubricant Gel, 20 g tube

FAHL® OPTIFLUID® Lubricant Gel, 3 g sachet

pack of 100

REF 36100

REF 36105