DE I. VORWORT

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EN I. FOREWORD

These instructions for use are valid for LARVYOX® Tweezers. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling and to ensure compliance with hygiene regulations. Please read the instructions for use carefully before using the product for the first time!

II. INTENDED USE

LARVYOX® Tweezers is a pair of encrustation Tweezers that are used to remove encrustations which have formed on the tracheostoma. The product is intended for domestic use only.

III. WARNINGS

The product is a single-patient product and is thus intended exclusively for one-time or repeated use in one individual - for example - patient. The user must first have been trained by qualified personnel in the safe handling and application. Application should always take place in front of a mirror. LARVYOX® Tweezers must not be inserted too deeply into the tracheostoma.

Cleaning agent and disinfectant residues can lead to irritations of the mucous membranes or other health impairments. The products of the cleaning agent and disinfectants can lead to irritation of the mucous membranes or other health impairments.

IV. PRODUCT DESCRIPTION

LARVYOX® Tweezers are angled, enabling you to have an unobstructed view in front of the mirror in order to remove the encrustations.

V. INSTRUCTIONS

Take LARVYOX® Tweezers out of the packaging, and clean and dry them before using them for the first time. Check whether the packaging and the product are in a faultless condition. Remove the encrustations on the inner edge of the tracheostoma bulge with the blunt front end of the LARVYOX® Tweezers, the end of the handle pointing downwards.

VI. SERVICE LIFE

Any LARVYOX® Tweezers that shows any damage, even if only minor, must be replaced immediately. LARVYOX® Tweezers that are presently not being used should be stored in a dry environment, protected from dust, direct sunlight and heat.

VII. CLEANING AND DISINFECTION

1. Cleaning
Clean LARVYOX® Tweezers with lukewarm water and a mild liquid soap, then rinse thoroughly with clear tap water and dry with a clean, lint-free cloth. In case of persistent soiling or contamination are present, LARVYOX® Tweezers can be treated by immersion in a special cleaning solution (OPTIBRUSH® CLEAN, Canülecleaning powder). After treatment, rinse thoroughly with clear tap water and dry with a clean, lint-free cloth. Make sure that there are no more cleaning agent residues on LARVYOX® Tweezers after cleaning.

Alternatively, cleaning by thermal disinfection is also possible. Place LARVYOX® Tweezers in boiling water (100 °C/212 °F) for at least 5 minutes, but not more than 10 minutes. Rinse LARVYOX® Tweezers several times with clear water.

2. Disinfection

Cold-disinfection with special chemical disinfectants is possible. Always clean according to the cleaning procedure described above before proceeding to disinfection (if applicable). For this purpose, OPTICIT® tube disinfectant should be used in accordance with the manufacturer's instructions. As an alternative, we recommend a disinfectant based on glutaraldehyde as active ingredient. Always observe the area of application and spectrum of activity specified for the disinfectant by the manufacturer. Follow the instructions for the disinfectant concerning the domaine of application et the spectre d'activité. Après à respecter la notice d'utilisation du désinfectant.

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