# LARCHEL® WASSERTHERAPIEGERÄT

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## PICTOGRAMM-LEGEND

---

 REF	Catalogue number		Keep away from sunlight
 LOT	Batch code		Manufacturer
	Consult instructions for use		Content (in pieces)
	Single patient use		

EN

## LARCHEL® WATER THERAPY DEVICE

---

### I. FOREWORD

These instructions for use apply to the LARCHEL® water therapy device. The instructions for use serve to inform the physician, authorised personnel and patient/user in order to ensure technically correct handling of the LARCHEL® water therapy device. **Please read the instructions for use carefully before using the product for the first time!**

Keep them in an easily accessible place for future reference.

Please also keep the instructions for use for as long as you use the tracheostomy tube. It contains important information on the product!

The LARCHEL® water therapy device is a medical device as defined by the MPG (German medical devices act) and conforms to, among others, the "Basic Requirements" as per Annex I of Directive 93/42 EEC.

Rehabilitation in water is one of the most important recovery processes following a serious illness or operation. But it is also well known that exercise in water contributes to general well-being. However, patients who have undergone a laryngectomy or tracheostomy have been denied this form of rehabilitation for a long time due to the lack of protective measures.

Not only does the larynx have the function of voice formation, it also protects the trachea by preventing food or liquids from entering the lungs unhindered. These natural protective mechanisms are lost after surgical removal of the larynx or performing a tracheostoma. Due to breathing through a permanent opening in the throat, there is a continuous risk of water entering unhindered into the trachea during swimming or bathing - the result would be drowning.

The LARCHEL® water therapy device was designed as a proven product to optimise the recovery process for patients after this type of operation and to allow them to continue participating in familiar activities of daily life such as swimming or bathing.

The LARCHEL® was developed at the end of the 60s by Paul V.G. Hauwaert. Following a severe illness and removal of the larynx, the inventor, who was an enthusiastic swimmer, was looking for a way to pursue his hobby. After years of experimenting, he succeeded in developing a device that enabled him to swim and breathe through his throat. He named his invention „LARCHEL®“, a name created by combining letters from the words laryngectomy and snorkel (Schnorchel in German).

The water therapy device LARCHEL® was first presented in Germany in 1974 at the annual conference of the German Society of ENT Medicine in Bad Reichenhall. The presentation turned out to be a complete success and the awareness level of the LARCHEL® increased steadily. Today, the LARCHEL® is a worldwide known product which has been in use successfully for many years.

In terms of design, selection of materials and features, the LARCHEL® has been further optimised over time to provide users with a reliable rehabilitation aid.

## II. INTENDED USE

The LARCHEL® water therapy device offers laryngectomised and tracheostomised patients the following application options:

- swimming
- aquatic therapy
- underwater massage
- smelling
- showering
- full bath

## III. INFORMATION ON ORDERING AND DELIVERY

We supply the LARCHEL® water therapy device REF 79160-XX with the following scope of delivery:

- 1 corrugated tube with mouthpiece incl. adapter
- 1 tracheostomy tube with cuff (TRACHEOSILC®)
- 1 cannula holder strap
- 1 10 ml syringe
- 1 separating wedge
- 1 towel
- 1 case
- 1 cleaning brush for the corrugated tube
- 1 instruction certification

The size specifications of the LARCHEL® refer to the sizes of the tracheostomy tube (see Instructions for Use TRACHEOSILC®).

Special accessory REF 79161:

- 1 snorkel with ring strap and elastic headband

### CAUTION!

**The LARCHEL® is only provided to authorised specialists, who have been specially trained and have proof of special training in the handling of the LARCHEL®, e.g., swimming representatives of laryngectomee associations. A doctor's prescription and confirmation of the introduction and handover must be presented for the recipient of the aid.**

## IV. CONTRAINDICATIONS

**Do not use if tracheostoma is unstable!**

**Physical and corporeal vitality as well as free airways are a basic prerequisite for the use of the water therapy device.**

**Use the LARCHEL® at the earliest 6 months after surgery or 3 months after radiation therapy. Significant premature use of the LARCHEL® can lead to complications such as infections.**

## V. WARNINGS

**The LARCHEL® is a single-patient product and thus intended exclusively for use in a single patient!**

The LARCHEL® water therapy device is a medical device.

The applicability of the LARCHEL® requires a detailed medical examination of the user and necessitates a medical prescription of the aid.

The LARCHEL® water therapy device may only be used after personal instruction of the user through authorised personnel (e.g. water therapy specialists).

**The LARCHEL® water therapy device is designed for use in fresh water. The use in salty water or water with ethereal additives leads to a higher material stress. This can impair the functionality of the LARCHEL®.**

Do not perform any type of adjustment, repair or alteration to the LARCHEL® water therapy device. If a product is damaged, it must be discarded as appropriate immediately!

Check the LARCHEL® water therapy device for visible damage before use!

Only use the LARCHEL® water therapy device with the tracheostomy tube included in the scope of delivery!

### VI. PRODUCT DESCRIPTION

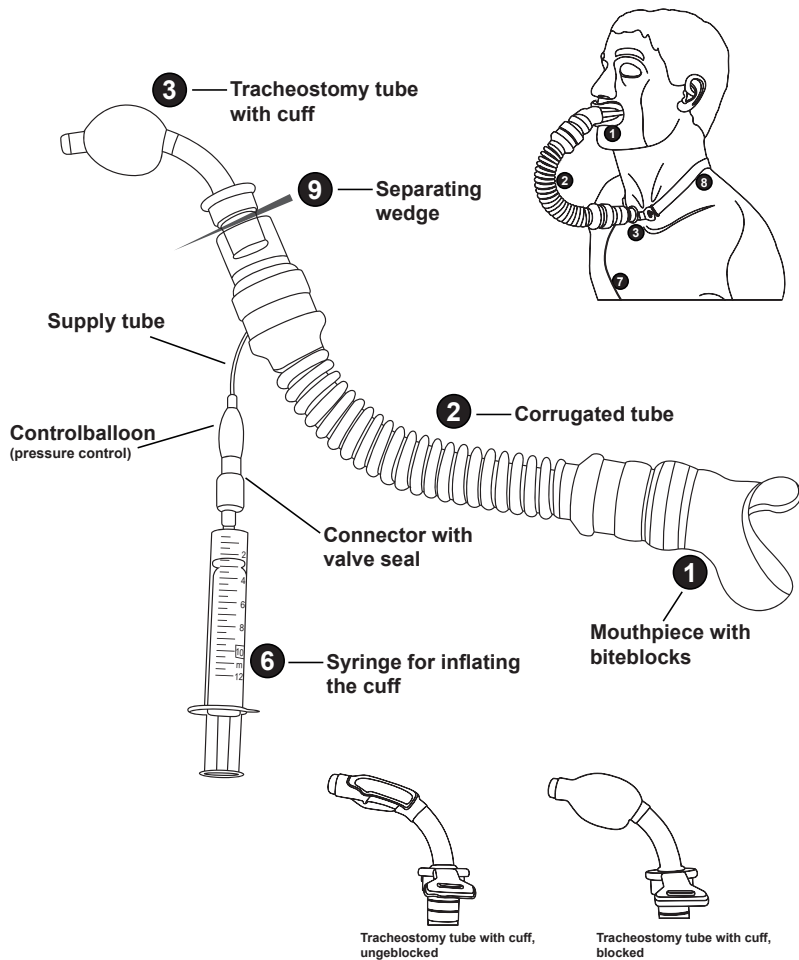
Figure 1

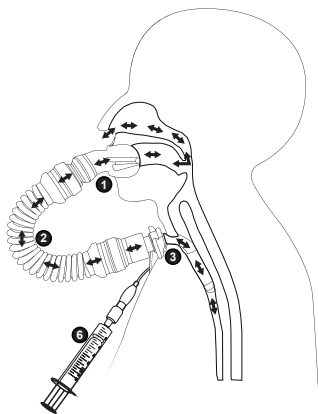
Figure 2

### (a) Application of the LARCHEL® water therapy device with mouthpiece

For use of the LARCHEL® water therapy device you require the following components:

- mouthpiece ❶
- corrugated tube ❷
- tracheostomy tube with cuff ❸
- disposable syringe ❹
- cannula holder strap ❺
- separating wedge ❻

The tracheostomy tube ❸ is located in the tracheostoma of the user (**Figures 1 and 2**). The mouthpiece ❶ is inserted into the mouth such that the lateral bite blocks are held between the teeth by biting with moderate pressure. The shield of the mouthpiece lies in front of the teeth and is firmly enclosed by the lips.

The air is now sucked in through the nose, channelled into the nose/pharynx, and then moves through the oral cavity to the corrugated tube ❷ with attached mouthpiece. The air is then channelled through the corrugated tube to the tracheostomy tube ❸ and flows into the tracheostoma. A tracheostomy tube with cuff is required to prevent the ingress of water into the tracheostoma.

#### **CAUTION!**

**Please observe the instructions for use of the enclosed tracheostomy tube with cuff!**

To exhale, the expelled air takes the same path in the opposite direction, i.e. the exhaled air reaches the tracheostomy tube ❸ via the lungs and trachea, is then passed to the mouth and nose/pharynx via the corrugated tube ❷ and is then exhaled again via the nose (**Figure 2**).

## (b) Application of the LARCHEL® water therapy device with snorkel

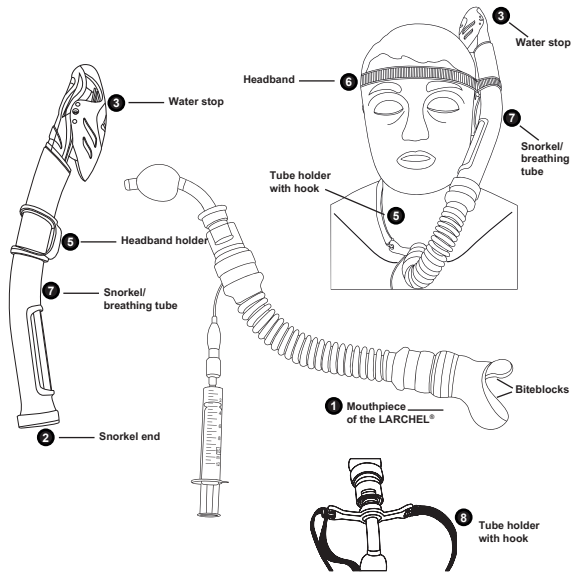
Figure 3

The snorkel (special accessory) is particularly helpful for those users who have difficulty breathing through the nose. In this case, it is essential to use a snorkel ⑦ (Figure 3) to ensure optimum airflow.

The snorkel ⑦ consists of a stable plastic tube, which is inserted approx. 1 - 2 cm deep into the corrugated tube. A special device at the end of the snorkel prevents any disconnection or ingress of water at the connection point.

A special attachment with a safety closing mechanism or water stop ③, is located at the upper end of the snorkel which prevents the unintentional ingress of water into the snorkel. The moveable flap in the housing of the attachment closes automatically when the water level is too high and blocks the opening to the snorkel. As a result, no water can enter the tracheostoma.

The riser of the snorkel is fitted with an elastic strap to stabilise the snorkel on the head and hold it in a vertical position ⑥.

During inhalation, air is taken in when the safety closing mechanism of the snorkel is open and passed directly into the tracheostomy tube of the tracheostoma via the snorkel and the corrugated tube. Exhalation takes place in the opposite direction.

## VII. INSTRUCTIONS

### (a) Preparation of the LARCHEL® water therapy device

The tracheostomy tube of the Larchel® water therapy device is available in various sizes. To achieve optimum sealing of the trachea, the exact cannula diameter must be selected. Consultation with the treating physician and the water therapy specialist is necessary here within the context of instructions!

1. Please check the scope of delivery for completeness.
2. Make sure that the tracheostomy tube is available in the appropriate cannula size. The outer diameter of the tracheostomy tube should correspond to the outer diameter of the cannula normally used by the user.

### CAUTION!

**As the sizing of the individual cannula manufacturers is based on different measuring principles, these cannot always be directly compared!**

3. Afterwards, a visual inspection of all device components including accessories (snorkel) must be performed.

**CAUTION!**

Never use the LARCHEL® water therapy device without instructions through authorised specialists!

Under no circumstances use the LARCHEL® if parts of the water therapy device or the snorkel are damaged!

**4. Assembly of the water therapy device**

**4.1 Application with mouthpiece:**

Connect the mouthpiece to the enclosed adapter and the corrugated tube (**Figure 4.a**). Check that the connection is secure by pulling.

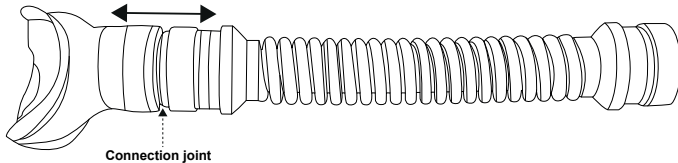
Figure 4.a

**4.2 Application with the snorkel:**

Remove the mouthpiece incl. adapter from the corrugated tube and connect it to the snorkel (**Figure 4.b**). Check for a secure connection by pulling. Connect the tracheostomy tube to the corrugated tube after completing 6. „When using a snorkel“.

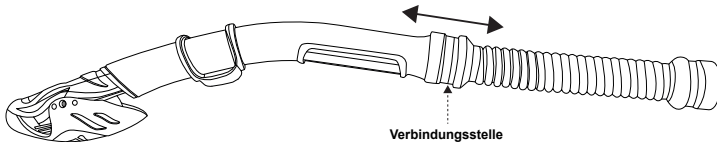
Figure 4.b

5. Insert the tracheostomy tube according to the instructions for use. For a secure hold of the tracheostomy tube, fix it to the side outlets of the cannula shield using the additional cannula holder strap with hook included in delivery, whereby the hooks of the cannula holder strap face outwards. Cough thoroughly before the next step. If you have not worn a tracheostomy tube for a longer period of time, a brief cough stimulation may occur.

**CAUTION!**

Observe the instructions for use of the tracheostomy tube!

The cannula holder strap may become loose in the water during use. Ensure that the strap is not too tight to prevent unintentional constriction of the neck. The holder strap must also not be too loose to prevent the tracheostomy tube from becoming detached unintentionally. Check the correct and secure fit of the cannula holder strap before use!

Stoma oil must only be used extremely sparingly. Under no circumstances must the oil reach the connection points of the LARCHEL®, as this increases the gliding properties of the connection points and there is a risk of disconnection during use! Caution when using local pain/anaesthetics and ointments for inflammation and pain in the area of the tracheostoma. The feeling for the correct fit and the correct sealing of the tracheostomy tube is affected by this!

## 6. When using a snorkel:

Now connect the corrugated tube to the tracheostomy tube. Block the cuff of the tracheostomy tube according to the corresponding instructions for use with the enclosed disposable syringe and check the pressure in the balloon using the control balloon.

## 7. Leak test

### 7.1 Leak test with mouthpiece and inserted tracheostomy tube:

To do this, connect the corrugated tube with the tracheostomy tube and the mouthpiece. Breathe in and briefly close the mouthpiece opening with your thumb as you exhale. During this procedure, no air must escape next to the cuff of the tracheostomy tube or from the connection points tracheostomy tube - corrugated tube or corrugated tube - mouthpiece.

### 7.2 Leak test with snorkel and inserted tracheostomy tube:

To do this, connect the corrugated tube with the tracheostomy tube and the snorkel. Hold the snorkel briefly with the safety closing mechanism facing downwards so that the water stop closes. Breathe in and the snorkel closes. During this procedure, neither air next to the cuff of the tracheostomy tube nor via the connection points tracheostomy tube - corrugated tube or corrugated tube - snorkel may penetrate into the closed system.

## CAUTION!

**If the leak test has not been passed, i.e. air escapes from the system or from the connection points, the LARCHEL® water therapy device must not be used!**

8. Perform breathing exercises to test the new airway (**Figures 1 and 2**): breathe in and out calmly and evenly several times as you take the mouthpiece into your mouth and enclose it tightly with your lips. Then try to breathe in through your nose. The tongue must not be placed on the palate so as not to block the air flow into the oral cavity and to the corrugated tube.

## CAUTION!

**The extension of the airway requires deeper inhalation and longer exhalation. Only use the LARCHEL® in water once you have mastered breathing in and out without any problems.**

### (b) Using the LARCHEL® in water

The first use of the LARCHEL® should always be accompanied and supervised by trained personnel (e.g. water therapy specialist) to monitor the correct handling of the LARCHEL®. Before entering the pool, shower your body to get used to the water temperature. Never swim with a full or completely empty stomach!

**For safety reasons, the water depth may only be selected such that the user can stand in the water at all times and the tracheostoma is above the water level when standing.**

The accompanying person should always be directly next to the user to be able to intervene immediately in case of an emergency.

Before entering the pool, the correct position of the LARCHEL® and the control balloon of the tracheostomy tube must be checked again. Only then is reliable sealing of the tracheostoma and thus protection against the ingress of water guaranteed. Take the mouthpiece of the LARCHEL® (**see VII (a)**) into your mouth and seal it tightly with your lips. The seal of the LARCHEL® must be checked by briefly immersing the upper part of the body in the water. To this purpose, hold on to the edge of the pool with one hand and use the accompanying person as support with the other hand. The user should then slowly kneel down until the upper body is immersed in the water and the tracheostoma is under the water level. If there is no coughing stimulus and no air bubbles rise, the tracheostoma is sufficiently sealed and the water therapy device is ready for use.

Swimming with the LARCHEL® requires a calm approach and careful movements in the water. Adaptation to this special aid is unfamiliar for many users, both physically and subjectively, due to the extended airway and increased muscle exertion.

Always swim in a prone position only and schedule your swimming time according to your physical condition.

Breathe slowly, deeply and vigorously as you swim. Avoid jerky movements to prevent the tracheostomy tube accidentally slipping out of the tracheostoma.

**CAUTION!**

**Never dive below the water level!**

**If you notice during swimming that there is not sufficient sealing, immediately raise your upper body out of the water. Do not panic. You can also hold your breath as a laryngectomised person and thus prevent water from entering your lungs. This will give you ample time to reach safety.**

**(c) Removing the LARCHEL®**

The LARCHEL® is easy to remove after completing the exercises or swimming in the water. If the snorkel was used, the headband should first be removed and the connection between snorkel and mouthpiece disconnected.

The syringe is used to unblock the cuff of the tracheostomy tube. To do this, place the unfilled syringe on the syringe attachment of the connector on the tracheostomy tube and withdraw the air from the cuff by slowly pulling the plunger of the disposable syringe. When the cuff is completely empty, the control balloon lies flat.

**CAUTION!**

**Do not attempt to pull the tracheostomy tube from the tracheostoma while the cuff is still filled with air, as this may result in significant tracheal injury.**

After unblocking the cuff, the cannula holder strap can be unhooked from the side eyelets of the cannula shield and the tracheostomy tube carefully removed from the tracheostoma.

**VIII. CLEANING AND STORAGE**

Immediately after using the water therapy device, the tracheostomy tube, adapter and corrugated tube with mouthpiece as well as the snorkel, in as far as used, should be thoroughly rinsed with warm tap water after disassembling the individual components. To this purpose, the tracheostomy tube can be removed quickly and easily from the corrugated tube with the included separating wedge. Use the included cleaning brush to clean the snorkel and the corrugated tube.

Due to the risk of infection, no secretion residues may remain in the LARCHEL® or the snorkel.

The accessible components of the LARCHEL® and snorkel should then be dried with a clean and lint-free cloth. Store the LARCHEL® dry in the appropriate transport case.

Also observe the instructions for use of the tracheostomy tube.

**Do not use any additives ( detergents ) for cleaning, as these could attack the material of the LARCHEL®.**

**CAUTION!**

**The LARCHEL® is a single patient product and is only intended for the exclusive use of one user!**

For storage purposes, we recommend wrapping the LARCHEL® in a clean and dry towel and protecting it from sunlight.

Use the supplied case to transport the LARCHEL®.

**IX. DURABILITY/PERIOD OF USE**

In the case of visible damage/material changes to the corrugated tubing, mouthpiece, snorkel or connectors, the water therapy device must no longer be used and be replaced with a new one!

If water enters the water therapy device via the corrugated tube, snorkel or connectors during application of the water therapy device, it must no longer be used and be replaced with a new one!

## X. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for functional deficiencies, injuries, infections, and/or other complications or other adverse events caused by unauthorised product alterations or improper use, care, and/or handling. Andreas Fahl Medizintechnik-Vertrieb GmbH is exempted from any liability including liability for defects - as far as legally permissible - when using the LARCHEL® water therapy device or in the case of use, application, care (cleaning, disinfection) in non-compliance with the instructions and specifications laid down in these instructions for use.

Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

Sale and delivery of all Andreas Fahl Medizintechnik-Vertrieb GmbH products is carried out exclusively in accordance with our General Terms and Conditions of Business (GTCs) which can be obtained directly from Andreas Fahl Medizintechnik-Vertrieb GmbH.

The manufacturer reserves the right to make changes to the product at any time without notice.

**LARCHEL®** is a trademark of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne, which is registered in Germany and the European Member States.

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