ES I. PRÓLOGO

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EN I. FOREWORD

These instructions for use are valid for LARYVOX® Adapt. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

Please read the instructions for use carefully before using the product for the first time!

II. INTENDED USE

LARYVOX® Adapt is an adapter with a 15 mm / 22mm connection.

III. WARNINGS

LARYVOX® Adapt is a single-patient product and is thus intended exclusively for one-time or repeated use in one individual patient.

Cleaning, disinfection, re-sterilisation and re-use can impair the safety and function of the product and are therefore not permissible!

IV. INSTRUCTIONS

LARYVOX® Adapt enables the connection of special aids (e.g. HME cassettes) with a 22 mm connector to a tracheostomy tube with a 15 mm connector ❶.

V. CLEANING

A mild, pH-neutral washing lotion can be used for cleaning.

After cleaning, rinse the LARYVOX® Adapt thoroughly several times with clear water and then dry it with a clean, soft, lint-free cloth.

The cleaning intervals for the adapters should be aligned towards the cleaning specifications applicable for the special aids connected. Please follow the respective instructions for use.

VI. SERVICE LIFE

LARYVOX® Adapt has an average expected life of 6 month with regular care. After that, the product must be replaced. The adapter has to be replaced immediately in case of any damage.

VII. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept an liability for damages especiall not for functional deficiencies, injuries, infections, and/or other complications or adverse events) caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use.

This applies - to the extent permitted by law - both to damages to the products themselves caused thereby and to any consequential damages caused thereby.

Product specifications described herein are subject to change without notice.

Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

LARYVOX® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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