









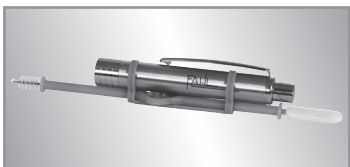
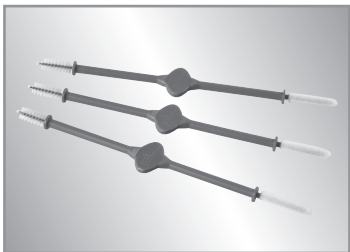
FAHL

LARYVOX® DUO BRUSH

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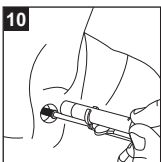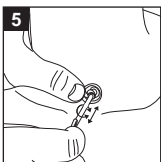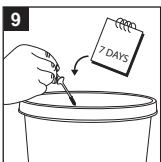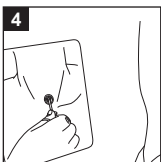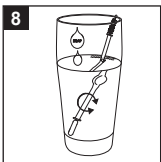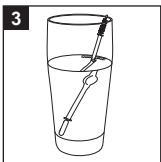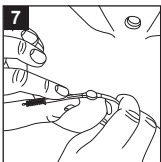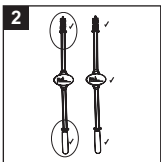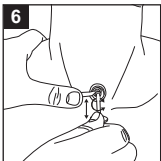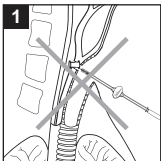INSTRUCTIONS FOR USE

I. VORWORT

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I. FOREWORD

These instructions for use are valid for the LARYVOX® Duo Brush cleaning brushes for voice prostheses. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

Please read the instructions for use carefully before using the product for the first time!

II. INTENDED USE

The cleaning brush is used for daily cleaning of the inner tube of the voice prosthesis.

III. WARNINGS

This product is a single-patient product and is thus intended exclusively for one-time or repeated use in one individual patient.

IV. COMPLICATIONS

Avoid incorrect introduction of the cleaning brush with tilting/slanting effects on the voice prosthesis, since this could cause dislocation of the voice prosthesis ❶.

V. CONTRAINDICATIONS

Do not use the LARYVOX® Duo Brush cleaning brushes for voice prostheses for a greatly widened shunt tube, because in such a situation secure retention of the voice prosthesis in the tissue is not assured.

VI. PRECAUTIONS

The correct brush size should be selected by the doctor/physician or the trained medical professionals.

Do not perform any type of adjustment, repair or alteration on brush or compatible accessories. If a product is damaged, it must be discarded immediately.

VII. PRODUCT DESCRIPTION

The cleaning brush has a rounded tip to prevent injuries to the mucous membranes of the oesophagus. By means of the sponge, residual secretions can be removed from the tracheal flange and its proximity. By means of the LARYVOX® Clip, a stoma lamp can be fixed to the cleaning brush in order to illuminate the voice prosthesis brightly for cleaning ❷. The LARYVOX® Stoma Light (REF 69101) can be ordered separately.

VIII. INSTRUCTIONS FOR CLEANING THE VOICE PROSTHESIS

Check the LARYVOX® Duo Brush cleaning brushes for voice prosthesis for possible damage before each use ❸. Only undamaged cleaning brushes should be used for cleaning the voice prosthesis.

Moisten the brush/sponge with clean water before use ❹.

Make sure your view is optimum ❺. If necessary, use the LARYVOX® Clip with a tracheostoma lamp in order to illuminate the voice prosthesis brightly.

Guide the cleaning brush gently into the inner tube of the voice prosthesis ❻. (If a replacement prosthesis is being used, it is then imperative to fix the safety thread of the voice prosthesis with at least one finger ❼).

Clean the voice prosthesis by gentle forward and back movements accompanied by gentle turning movements ❽/❾.

To optimise accessibility to the voice prosthesis, the cleaning brush may be bent individually at the flexible brush body (not at the brush wire!) ❿.

IX. CLEANING AND DISINFECTION

Clean the voice prosthesis at least 2 times daily with the cleaning brush.

After use, remove the residual secretions from the cleaning brush and/or the cleaning sponge with a clean cloth.

In addition, carefully rinse the cleaning brush and/or the cleaning sponge by swirling in a soap solution of liquid soap for 15 seconds ⑧.

Then rinse out with clear, almost sterile water.

Allow the cleaning brush to dry in the air.

Regular disinfection one time per day is recommended.

Especially in cases of potentially high risk of contamination (after contact with the floor or with pets, close proximity to patients with respiratory diseases or infections, and in principle in a hospital environment, where the risk of infection is high on the whole), the cleaning brush should be disinfected before as well as after each use.

A cleaning brush may be disinfected with the following substances: 70% ethanol for 10 minutes, 70% isopropyl alcohol for 10 minutes, 3% hydrogen peroxide for 60 minutes.

X. STORAGE/CARE

Store the dry cleaning brush hygienically in the associated case.

XI. SERVICE LIFE

We recommend replacing the cleaning brush one time per week ⑨.

XII. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for functional deficiencies, injuries, infections, and/or other complications or other adverse events caused by unauthorised product alterations or improper use, care, and/or handling.

If the LARYVOX® Duo Brush cleaning brushes for voice prostheses are used beyond the period of use specified under Section XI, and/or if it is used, maintained (cleaned, disinfected) in non-compliance with the instructions and specifications laid down in these instructions for use, Andreas Fahl Medizintechnik-Vertrieb GmbH will, as far as legally permissible, be free of any liability, including liability for defects.

Should a serious adverse event occur in connection with this product of Andreas Fahl-Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.

Sale and delivery of all Andreas Fahl Medizintechnik-Vertrieb GmbH products is carried out exclusively in accordance with our General Terms and Conditions of Business which can be obtained directly from Andreas Fahl Medizintechnik-Vertrieb GmbH.

Product specifications described herein are subject to change without notice.

LARYVOX® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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