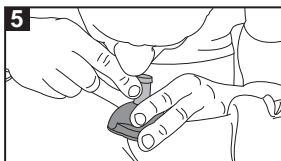
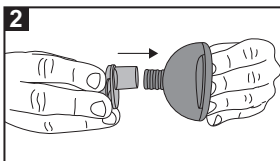
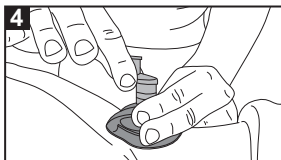
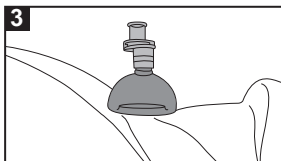
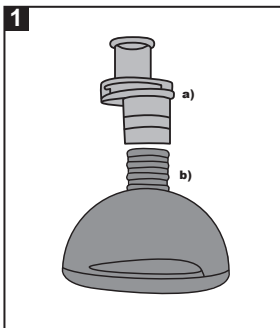
LARYVOX® SOS SAFE-MASK

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I. FOREWORD

These instructions for use are valid for LARYVOX® SOS safe-mask.

The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

II. INTENDED USE

LARYVOX® SOS safe-mask comprises LARYVOX® mask (a) and a one-way valve (b) ❶. LARYVOX® SOS safe-mask enables mouth-to-tracheostoma ventilation in emergency situations. The one-way valve conducts the exhalation flow/ secretions away from the person providing aid.

III. WARNINGS

At change of patient, the mask must either be replaced, or reprocessed and sterilised in a technically correct manner. Infection hazard!

IV. INSTRUCTIONS

1. Fit the one-way valve onto the connection cone of the LARYVOX® mask ❷.
2. Place the LARYVOX® SOS safe-mask directly on the tracheostoma or on the tracheostomy tube ❸.
3. Grip the one-way valve and press the mask against the skin until complete sealing is achieved ❹.
4. Blow air into the valve with your mouth. If rescue breathing is successful, the chest will rise ❺.
5. Continue ventilation (12x per minute)

CAUTION

Heed the general first aid measures!

Immediately alert the emergency physician/ emergency medical service!

V. SERVICE LIFE

If the material has become discoloured or damaged, the tracheostoma mask must be replaced.

VI. CLEANING, DISINFECTION AND STERILISATION

For reasons of hygiene, the LARYVOX® mask must be cleaned after each use. Rinse the LARYVOX® mask thoroughly under lukewarm running water.

If necessary, the LARYVOX® mask can additionally be disinfected with a suitable disinfectant. When doing this, the disinfectant manufacturer's instructions for use must be observed. After disinfection, rinse with clear water to avoid contact of the patient with the disinfectant.

As required in each individual case, the LARYVOX® mask can be thermally disinfected, i.e. disinfected by boiling (for at least 5 minutes in hot water at 100 degrees Celsius/212 degrees Fahrenheit). A time period of 10 minutes should not be exceeded.

CAUTION

It is not permissible to clean the one-way valve in boiling water.

Allow the LARYVOX® mask to cool down before using it again to avoid scalds and burns.

The LARYVOX® mask can be autoclaved or sterilised with ethylene oxide (EO) using validated techniques.

VII. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages (especially not for functional deficiencies, injuries, infections, and/or other complications or adverse events) caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use. This applies – to the extent permitted by law – both to damages to the products themselves caused thereby and to any consequential damages caused thereby.

Product specifications described herein are subject to change without notice.

LARYVOX® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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