## DE I. VORWORT

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## EN I. FOREWORD

These instructions for use are valid for tube holders.  
Please read the instructions for use carefully before using the product for the first time!

## II. INTENDED USE

The suffix PED identifies the tracheostomy tube to be fixed securely in place and can be adjusted to fit different necksizes.  
The suffix PED identifies products for paediatric use.

## III. WARNINGS

**NECKFIX<sup>®</sup> FIMES** are single-patient products and are intended for single use only.

## CAUTION

Cleaning, disinfection, re-sterilisation and re-use can impair the safety and function of the product and are therefore not permissible!  
The tubeholder should not be fitted too tightly around the neck. There should ideally be a distance of approx. 1 cm between tube holder and neck.  
To prevent the tracheostomy tube from coming off by accident, the tube holder must be checked regularly to make sure it is still securely fixed in place.

## IV. INSTRUCTIONS

**NECKFIX<sup>®</sup> DUO**  
Attach the two parts of the tube holder to each other by means of the adherent fastener, so that the correct length (neck size) is obtained (► a).

**NECKFIX<sup>®</sup> DUO**  
Place the tube holder around your neck with the hook-and-loop fasteners facing outward (► b). Attach the two hook-and-loop fasteners to the lateral eyelets on both sides of the neck flange (► c).

## V. SERVICE LIFE

The maximum period of use / wearing period is 24 hours.

## VI. LEGAL NOTICE

The manufacturer Andreas Fahrl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use. Should serious adverse events occur in connection with this product of Andreas Fahrl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled. Product specifications described herein are subject to change without notice. **NECKFIX<sup>®</sup>** is a trademark and brand of Andreas Fahrl Medizintechnik-Vertrieb GmbH. Weitere registrieren in Germany and the full member states.

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Andreas FahI Medizintechnik-Vertrieb GmbH