DE I. VORWORT

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EN I. FORWARD

These instructions for use are valid for tube holders.

The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure correct handling.

Please read the instructions for use carefully before using the product for the first time!

II. INTENDED USE

The tube holder allows the tracheostomy tube to be fixed securely in place and can be adjusted to fit different neck sizes.

The suffix „PED“ identifies products for paediatric use.

III. WARNINGS

The tube holder is a single-patient product and is thus intended exclusively for use - including repeated use - in one and the same patient.

The tube holder should not be fitted too tightly around the neck. There should ideally be a distance of approx. 1 cm between tube holder and neck.

To prevent the tracheostomy tube from coming off by accident, the tube holder must be checked regularly to make sure it is still securely fixed in place.

IV. INSTRUCTIONS

NECKFIX® DUO
Attach the two parts of the tube holder to each other by means of the adherent fastener, so that the correct length (neck size) is obtained. **➊**

NECKFIX®/NECKFIX® DUO
Place the tube holder around your neck with the hook-and-loop fasteners facing outward. **➋** b. Attach the two hook-and-loop fasteners to the lateral eyelets on both sides of the neck flange. **➌**

V. SERVICE LIFE

The service life depends on the individual frequency of use and on individual handling. The products must be replaced immediately when damaged.

VI. LEGAL NOTICES

The manufacturer Andreas Fahrl Medizintechnik-Vertrieb GmbH will not accept any liability for damages caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use. Should a serious adverse event occur in connection with this product of Andreas Fahrl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled. The product and brand names are subject to change without notice. NECKFIX® is a trademark and brand of Andreas Fahrl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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