
KACLIP®

GBRUCHSANWEISUNG • INSTRUCTIONS FOR USE

DE KANÜLENTRAGEBAND	HU KANÜLÉRTARTÓ SZALAG
EN TUBE HOLDER	PL TASMA STABILIZACJA RURKIĘ
FR BANDE DE FIXATION DE CANULE	RU ЛЕНТА ДЛЯ ФИКСАЦИИ КАНОИИ
IT FASCIA DI FISSAGGIO CANNULA	CS POPRUH KANLY
PT FAIXA DE FIXAÇÃO DE A CÁNULA	SK PÁS NA NOSENIE KANVLY
ES CINTA DE FIXAÇÃO DA CÁNULA	SL TRAK ZA PRICRŠTITEV KANILE
NL CANULEDRAGBAND	HR FIKSACIONA VRPKA ZA KANULU
SV KANÜLYBAND	HR POVEZNA TRAKA ZA KANULU
DA KANYLEBÆREBAND	BG ЗАКРЕПВАЩА ЛЕНТА ЗА КАНОЛИАТА
NO BÆRESTOPP FOR KANYLE	RO BANDE DE SUSȚINERE A CANULEI
FI KANYYLINTUKIHOIHNNA	JA カニューライト
EL ΤΑΜΙΑ ΣΤΕΡΕΩΣΗΣ ΤΡΑΧΕΙΩΣΗ/ΗΝΑ	AR شريط تحلل ايبس
TR KANÜL BİYERİN BAĞI	

BILDER / PICTURES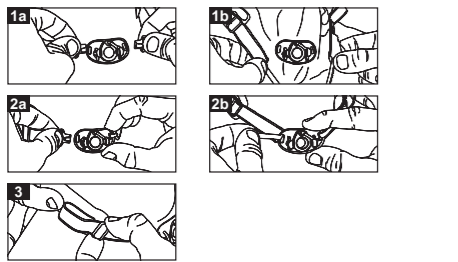

KACLIP® HM, BEIGE

Länge / Length 30 – 50 cm / 11.81 – 19.69"

REF 3205	LOT 20210115
----------	--------------

0140451948053890
1020210115

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EN I. FOREWORD

These instructions for use are valid for tube holders. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. Please read the instructions for use carefully before using the product for the first time!

II. INTENDED USE

The tube holder allows the tracheostomy tube to be fixed securely in place and can be adjusted to fit different neck sizes.

III. WARNINGS

The tube holder is a single-patient product and is thus intended exclusively for use - including repeated use - in one and the same patient. The tube holder should not be fitted too tightly around the neck. There should be a distance of approx. 1 cm between the tube holder and the neck. To prevent the tracheostomy tube from coming off by accident, the tube holder must be checked regularly to make sure it is still securely fixed in place.

IV. INSTRUCTIONS

Place the tube holder around your neck with the hooks/hook-and-loop fasteners facing outward. Attach the two hooks/hook-and-loop fasteners to the lateral eyelets on both sides of the neck flange. Adjust the slide on the tubeholder in accordance with your individual neck size.

V. CLEANING

Hooks and hook-and-loop fasteners must be fastened prior to washing. The tube holder may only be washed by hand up to 30 degrees.

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