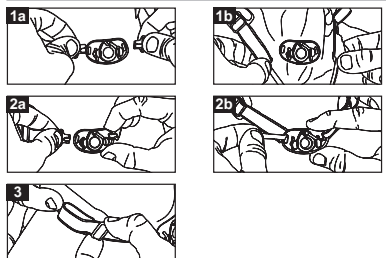
SUPRAFIX® 15 H PED

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EN I. FOREWORD

The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. Please read the instructions for use carefully before using the product for the first time.

II. INTENDED USE

The tube holder allows the tracheostomy tube to be fixed securely in place and can be adjusted to fit different necksizes.
CAUTION!
The tube holder is a single-patient product and is thus intended exclusively for use - including repeated use - in one and the same patient. The tubeholder should not be fitted too tightly around the neck. The tube holder should be adjusted to fit between the neck size and the tube holder and neck. To prevent the tracheostomy tube from coming off by accident, the tube holder must be checked regularly to make sure it is still securely fixed in place.

III. INSTRUCTIONS

Place the tube holder around your neck with the hooks/hook-and-loop fasteners facing outward 0/a/b. Attach the two hooks/hook-and-loop fasteners to the side eyes at both sides of the neck flange 0/a/b. Adjust the slide on the tubeholder in accordance with your individual neck size 0/d.

IV. SERVICE LIFE

The service life depends on the individual frequency of use and on individual handling. The products must be replaced immediately when damaged.

V. LEGAL NOTICES

The manufacturer Andreas FahI Medizintechnik-Vertrieb GmbH accepts no liability for damage caused by unauthorised product modifications, repairs not carried out by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products contrary to the provisions of these instructions for use.

Should a serious adverse event occur in connection with this product of Andreas FahI Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled. The manufacturer reserves the right to make changes to the product at any time without notice. SUPRAFIX® is a trademark of Andreas FahI Medizintechnik-Vertrieb GmbH, Cologne, which is registered in Germany and the European Member States.

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