DE I. VORWORT

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EN I. FOREWORD

These instructions for use are valid for OPTIBRUSH® SWAB. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technicaly correct handling. **Please read the instructions for use carefully before using the product for the first time!**

II. INTENDED USE

This non-sterile cleaning swab is intended for gentle cleaning of tracheostomy tubes.

III. WARNINGS

OPTIBRUSH® SWAB is a single-patient product and is intended for single use only. Always remove the swab directly into the tracheostoma before commencing cleaning. Never insert the cleaning swab directly into the tracheostoma!

IV. INSTRUCTIONS

Moisten the swab/sponge with clean water before use **●**. Remove the tracheostomy tube from the tracheostoma **●**. Clean the tracheostomy tube by slight forward/backward movements accompanied by turning movements **●**.

V. LEGAL NOTICES

The manufacturer Andreas Fahl Medzintechnik-Vertrieb GmbH accepts no liability for damage caused by unauthorised product modifications, repairs not carried out by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products contrary to the provisions of these instructions for use. Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled. The manufacturer reserves the right to make changes to the product at any time without notice. OPTIBRUSH® is a trademark of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne, which is registered in Germany and the European Member States.

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