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EN I. FOREWORD
These instructions for use are valid for OPTIBRUSH® canula cleaning brush.
The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.
Please read the instructions for use carefully before using the product for the first time!
II. INTENDED USE
The OPTIBRUSH® is a special brush for gentle cleaning of tracheal cannulas.
III. WARNINGS
Always remove the cannula from the tracheostoma before commencing cleaning. Never insert the cannula cleaning brush directly into the tracheostoma!
CAUTION!
OPTIBRUSH® is a single-patient product.
IV. CAUTION
Do not repair or modify the brush. Damaged products need to be properly disposed immediately.

V. INSTRUCTIONS
The OPTIBRUSH® should be reshaped prior to initial use according to the shape of the cannula tube.
In order to avoid damage, it is recommended to clean the tracheal cannula with the brush starting at the cannula tip. This serves to simplify handling.
VI. CLEANING
For hygienic reasons, the cannula cleaning brush should be thoroughly rinsed with clear water after each use and subsequently placed in a beaker or glass for drying with the handle pointing down.
VII. SERVICE LIFE
Depending on frequency and level of use, we recommend replacing the cannula cleaning brush on a regular basis. This is especially important when the bristles are worn, discolored or stuck together.
VIII. LEGAL NOTICES
The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH accepts no liability for damage caused by unauthorised product modifications, repairs not carried out by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products contrary to the provisions of these instructions for use. Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.
The manufacturer reserves the right to make changes to the product at any time without notice.
OPTIBRUSH® is a trademark of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne, which is registered in Germany and the European Member States.

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