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EN
I. FOREWORD
These instructions for use are valid for tracheal compresses. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

II. INTENDED USE
Tracheal compresses serve to absorb tracheal secretions and to cushion the tracheal neck flange against the skin.

III. WARNINGS
Should you experience skin irritation (e.g. redness, itchiness), discontinue the product immediately and consult a physician. This may be due to special skin sensitivity or allergy-related incompatibility.

CAUTION!
Cleaning, disinfection, re-sterilisation and re-use can impair the safety and function of the product and are therefore not permissible!

IV. PRODUCT DESCRIPTION

Different versions of tracheal compresses are available. The compresses have a circular hole through which the cannula tube can be passed and are also available with an additional slit opening, for fast compress changing.

V. INSTRUCTIONS

Before replacing the compress, ensure that you have all required material available. The tracheal cannula should be cleaned and if required, disinfected before application. The tracheal cannula is inserted through the existing, circular opening of the compress **⊕**. The skin-side of the compress is furnished with a turquoise thread.

SENSOTRACH[®] ALU SLIT/ SENSOTRACH[®] UNO SLIT/ SENSOTRACH[®] UNO PED SLIT/ SENSOTRACH[®] UNO PED ALU SLIT/ SENSOTRACH[®] UNO PED SLIT Junior/ SENSOTRACH[®] DUO SLIT/ SENSOTRACH[®] CH DUO ALU/ SENSOTRACH[®] DUO PED SLIT/ SENSOTRACH[®] DUO PED SLIT/ SENSOTRACH[®] DUO ALU SLIT/ SENSOTRACH[®] SKIN SLIT/ SENSOTRACH[®] 3-Plus SLIT:
An additional slit next to the circular opening allows fast compress replacement without prior removal of the tracheal cannula from the tracheostoma **⊕**.

SENSOTRACH[®] ALU/ SENSOTRACH[®] ALU SLIT/ SENSOTRACH[®] UNO PED ALU SLIT/ SENSOTRACH[®] DUO ALU/ SENSOTRACH[®] DUO ALU SLIT:
The skin-side of the compress is vapour deposited with aluminium.

CAUTION!

When inserting the tracheal cannula, please observe the application instructions of its manufacturer.

VI. SERVICE LIFE

The compress is unsterile and intended for single use only. The maximum period of use / wearing period is 24 hours.

VII. LEGAL NOTICES

The manufacturer Andreas FahI Medizintechnik-Vertrieb GmbH does not accept any liability for damages (especially not for functional deficiencies, injuries, infections, and/or other complications or adverse events) caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use. This applies – to the extent permitted by law – both to damages to the products themselves caused thereby and to any consequential damages caused thereby. Should a serious adverse event occur in connection with this product of Andreas FahI Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled. Product specifications described herein are subject to trademark and patent law. SENSOTRACH[®] is a trademark and a brand of Andreas FahI Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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