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EN

I. FOREWORD

These instructions for use are valid for SENSOTRACH® Foam.

The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

Please read the instructions for use carefully before using the product for the first time!

II. INTENDED USE

SENSOTRACH® Foam is used, in combination with a tracheostomy tube, to align the skin level of an irregular tracheostoma that is situated below the skin level.

The specific option is improved adhesion and reduced losses of air.

SENSOTRACH® Foam is available in different versions:

a) SENSOTRACH® Foam OVAL (REF 30790) SENSOTRACH® Foam PRO OVAL (REF 30790)

b) SENSOTRACH® Foam ROUND (REF 30791) SENSOTRACH® Foam PRO ROUND (REF 30792)

The different versions (ROUND 🇬🇧 / OVAL 🇬🇧) serve for customisation to achieve a uniform skin level. To accomplish this, the variants can be used in combination 🇬🇧.

III. WARNINGS

SENSOTRACH® Foam are single-patient products and are intended for single use only.

CAUTION! Cleaning, disinfection, re-sterilisation and re-use can impair the safety and function of the product and are therefore not permissible!

It must not be cleaned or disinfected, as use of cleaning agent can render SENSOTRACH® Foam unusable.

The SENSOTRACH® Foam must under no circumstances be inserted directly into the tracheostoma!

Should you experience skin irritation (e.g. redness, itchiness), discontinue the product immediately and consult a physician. This may be due to special skin sensitivity or allergy-related incompatibility.

IV. COMPICATIONS

Due to the specific adhesive properties of SENSOTRACH® Foam PRO, reversible skin irritations can occur in individual cases.

V. CONTRAINDICATIONS The use of SENSOTRACH® Foam PRO should be avoided during radiotherapy, due to the risk of skin irritations.

After undergoing radiotherapy, only use SENSOTRACH® Foam PRO after consulting your treating physician. Start using the tape at the earliest 2 weeks after completing radiotherapy.

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VI. INSTRUCTIONS CAUTION!

The tracheal cannula should be cleaned and if required, disinfected before application. Please observe the manufacturer's instructions with regard to cleaning, disinfection and use of the adhesive on the cannula.

a) SENSOTRACH® Foam

The tracheal cannula is inserted through the existing, circular opening of the compress 🇬🇧.

b) SENSOTRACH® Foam PRO Clean the skin around the tracheostoma in front of a mirror using warm water and pH-neutral soap or OPTIFAH®[®] stoma cleaning cloths. Dry the skin. Before cleaning, remove residues of adhesive from the skin if necessary, using OPTICLEAR®[®].

To protect the skin from irritation, apply OPTIGARD®[®] to the affected area of the skin.

Remove the protective foil on the rear side 🇬🇧. The tracheostomy tube is inserted through the circular opening provided 🇬🇧. This is done with the adhesive surface facing toward the tracheostoma.

CAUTION! Do not use creams that contain fat! Otherwise, secure attachment is no longer ensured.

VII. SERVICE LIFE

The use of / wearing period is 24 hours.

Deviations from the expected wearing period occur as a result of individual wearing conditions and skin properties.

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