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**EN I. FOREWORD**

These instructions for use are valid for tracheal compresses. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

**II. INTENDED USE**

Tracheal compresses serve to absorb tracheal secretions and to cushion the tracheal neck flange against the skin.

**III. WARNINGS**

Should you experience skin irritation (e.g. redness, itchiness), discontinue the product immediately and consult a physician. This may be due to special skin sensitivity or allergy-related incompatibility.

**IV. PRODUCT DESCRIPTION**

Different versions of tracheal compresses are available. The compresses have a circular hole through which the cannula tube can be passed and are also available with an additional slit opening, for fast compress changing.

**V. INSTRUCTIONS**

Before replacing the compress, ensure that you have all required material available. The tracheal cannula should be cleaned and if required, disinfected before application. The tracheal cannula is inserted through the existing, circular opening of the compress **!**

**CAUTION!**

When inserting the tracheal cannula, please observe the application instructions of its manufacturer.

**VI. SERVICE LIFE**

The compress is unsterile and intended for single use only. The usage period depends on the strength of the secretion.

**VII. LEGAL NOTICES**

The manufacturer Andreas FahI Medizintechnik-Vertrieb GmbH accepts no liability for damage caused by unauthorised product modifications, repairs not carried out by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products contrary to the provisions of these instructions for use. Should a serious adverse event occur in connection with this product of Andreas FahI Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled. The manufacturer reserves the right to make changes to the product at any time without notice. SENSOTRACH® is a trademark of Andreas FahI Medizintechnik-Vertrieb GmbH, Cologne, which is registered in Germany and the European Member States.

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