
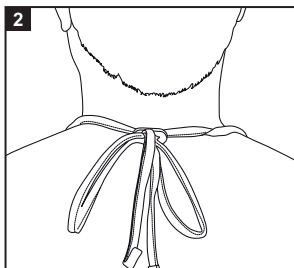
BUCHANAN® BIB S

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INSTRUCTIONS FOR USE

BILDER / PICTURES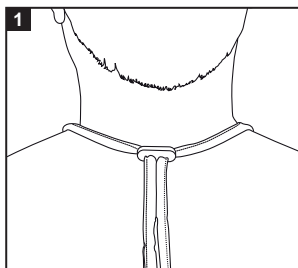

PIKTOGRAMM-LEGENDE / PICTROGRAM LEGEND

Hersteller /
Manufacturer

Gebrauchsanweisung beachten /
Consult instructions for use

Bestellnummer /
Catalogue number

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Inhaltsangabe in Stück /
Content (in pieces)

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I. FOREWORD

These instructions are valid for **BUCHANAN® BIB S (15*12 cm)**. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

II. INTENDED USE

BUCHANAN® BIBs are designed to be used for covering the tracheostoma and prevent the intrusion of foreign objects into the airways. At the same time, the breathing air is filtered, warmed and humidified, and escaping tracheal secretions are absorbed.

III. WARNINGS

BUCHANAN® BIBs are single-patient products and are thus intended exclusively for one-time or repeated use in one individual patient.

It is not permissible to machine-wash, tumble-dry or bleach the **BUCHANAN® BIB**.

The **BUCHANAN® BIB** should not be washed more than three times. If the bib is washed very often, the foam might lose its strength and precipitate.

IV. PRECAUTIONS

If the **BUCHANAN® BIB** bibs are damaged or show signs of wear (e.g. fraying or the like), they must be replaced! If the product is damaged, its function can be impaired!

The **BUCHANAN® BIB** must not be worn too tightly around the neck. Approx. 1 cm clearance is ideal.

VI. PRODUCT DESCRIPTION

BUCHANAN® BIB are placed around the neck and completely cover the tracheostoma. They must not be worn too tightly around the neck. Approx. 1 cm clearance is ideal.

VII. APPLICATION

Place the fabric straps of the **BUCHANAN® BIB** around the neck and pull the adjuster with a distance of approx. 1 cm to the neck ❶. Now knot the fabric straps with a bow ❷.

VIII. SERVICE LIFE

BUCHANAN® BIB must be changed every day for reasons of hygiene. They are intended for repeated use and should not be washed more than three times. The products have to be replaced immediately in case of any damage.

IX. CLEANING

Wash **BUCHANAN® BIB** at 30°C by hand using mild detergents only.

It is not permissible to machine-wash, tumble-dry or bleach the **BUCHANAN® BIB**.

Smooth the **BUCHANAN® BIB** and allow to dry between two towels. This avoids creasing of the foam layer. **IX. LEGAL NOTICES**

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages (especially not for functional deficiencies, injuries, infections, and/or other complications or adverse events) caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use.

This applies – to the extent permitted by law – both to damages to the products themselves caused thereby and to any consequential damages caused thereby.

Product specifications described herein are subject to change without notice.