DE VORWORT

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EN I. FOREWORD

These instructions are valid for LARYTAPE®.

The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

II. INTENDED USE

LARYTAPE® filter the breathing air, thereby reducing intrusion of particles into the patient's airways.

They store the moisture and warmth of the exhaled air in the filter medium and emit these to the inhaled air again during inspiration.

We supply LARYTAPE® in various sizes. With a button LARYTAPE® TOUCH enables closure of the tracheostoma, and hence phonation.

III. WARNINGS

LARYTAPE® are single-patient products and are intended for single use only.

CAUTION!

Cleaning, disinfection, re-sterilisation and re-use can impair the safety and function of the product and are therefore not permissible!

Should you experience skin irritation (e.g. redness, itchiness), discontinue the product immediately and consult a physician. This may be due to special skin sensitivity or allergy-related incompatibility.

IV. INSTRUCTIONS

Clean the skin around the tracheostoma in front of a mirror using warm water and pH-neutral soap or OPTIFAHL® stoma cleaning drops. Dry the skin. Before cleaning, remove residues of adhesive from the skin if necessary with OPTICLEAR®. After removing the protective foil backing ② place the tracheostoma protection centred over the tracheostoma ① and cover the tracheostoma this way.

Remove the tracheostoma protection ② by carefully removing the adhesive strip from the skin.

LARYTAPE® TOUCH:

For speaking, press the button down slightly with your finger so as to enable phonation by means of the closure ④.

CAUTION!

Do not use creams that contain fat!

Otherwise, secure attachment is no longer ensured.

V. SERVICE LIFE

The maximum period of use/wearing period is 24 hours.

The tracheostoma protection can also be changed several times a day, as necessary.

VI. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH accepts no liability for damage caused by unauthorised product modifications, repairs not carried out by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products contrary to the provisions of these instructions for use.

Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled. The manufacturer reserves the right to make changes to the product at any time without notice.

LARYTAPE® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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