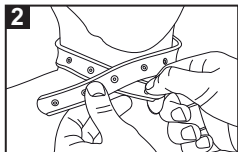
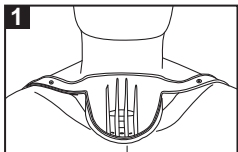
SECUTRACH® DUSCHESCHUTZ / SHOWER GUARD

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SECUTRACH® SHOWER GUARD

I. FOREWORD

These instructions are valid for the **SECUTRACH® shower guard**. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. **Please read the instructions for use carefully before using the product for the first time!**

II. INTENDED USE

The **SECUTRACH®** serves to protect the tracheostoma while showering and washing hair. The **SECUTRACH®** largely prevents water entering the tracheostoma. As a 100% seal cannot be achieved, special caution is advised in use.

III. WARNING

The **SECUTRACH®** is a single-patient product and is thus intended exclusively for one-time or repeated use in one individual patient. Heating to over 65°C, disinfection with boiling water or steam sterilisation are not allowed and lead to damage of the **SECUTRACH®**. Never hold the water jet directly on the tracheostoma.

IV. PRODUCT DESCRIPTION

The **SECUTRACH®** is tied around the neck and closed with a button fastener. It is adjustable for all neck sizes.

V. CONTRAINDICATION

The **SECUTRACH®** serves exclusively to protect the tracheostoma while showering and washing hair. It offers no protection while bathing if the tracheostoma is inadvertently immersed under water.

CAUTION! The SECUTRACH® must never be used with a full bath and for water sports.

VI. COMPLICATIONS

Entry of water into the tracheostoma, even just a small amount, cause an urge to cough through the natural protective reflexes. In this event, the user has to leave the shower immediately or immediately interrupt the process of hair washing. The **SECUTRACH®** must then be repositioned.

VII. INSTRUCTIONS

Tie the **SECUTRACH®** around the neck so that the air openings are positioned below the tracheostoma ❶. The air openings are located in the lower area of the **SECUTRACH®** rubber cap. The air flows in through it. Close the **SECUTRACH®** with the aid of the button fastener so it seals tightly to the skin ❷.

VIII. CLEANING

The **SECUTRACH®** must be cleaned thoroughly after each use to prevent functional impairment to the **SECUTRACH®** and to minimise the risk of infection. A mild, pH-neutral washing lotion can be used for cleaning. After cleaning, rinse the **SECUTRACH®** thoroughly several times with clear water and then dry it with a clean, soft, lint-free cloth. Do not use any sharp-edged cleaning utensils!

IX. SERVICE LIFE

Do not use the **SECUTRACH®** if it is damaged or if there are residues/soiling that can no longer be removed by rinsing.

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