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EN **I. FOREWORD**
These instructions for use are valid for LARYNGOFIX® / TRACHEOFIX / TRACHEOSTOMA-FIX® tracheostoma protection.
The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.
Please read the instructions for use carefully before using the product for the first time!
II. INTENDED USE
LARYNGOFIX® / TRACHEOFIX / TRACHEOSTOMA-FIX® filter the breathing air, thereby reducing the risk of particles entering the respiratory tract.
They store the moisture and warmth of the exhaled air in the filter medium and emit these to the inhaled air again during inspiration.
We supply LARYNGOFIX® / TRACHEOFIX / TRACHEOSTOMA-FIX® in various sizes.
With a button LARYNGOFIX® TOUCH enables closure of the tracheostoma, and hence phonation.

III. WARNINGS
LARYNGOFIX® / TRACHEOFIX / TRACHEOSTOMA-FIX® are single-patient products and are intended for single use only.
It must not be cleaned or disinfected, as use of cleaning agent can render the filter medium unusable.
Sterilization and reuse can impair the function and is therefore not permissible either.
Should you experience skin irritation (e.g. redness, itchiness), discontinue the product immediately and consult a physician. This may be due to special skin sensitivity or allergy-related incompatibility.

IV. INSTRUCTIONS
Clean the skin around the tracheostoma in front of a mirror using warm water and pH-neutral soap or OPTIFAHIL® skin cleaning cottons. Dry the skin. Before cleaning, remove residues of adhesive from the skin if necessary, using OPTICLEAR®.
After removing the protective foil backing **●**, place the tracheostoma protection centred over the tracheostoma **●** and cover the tracheostoma this way **●**.
Remove the tracheostoma protection **●** by carefully removing the adhesive strip from the skin.

LARYNGOFIX® TOUCH:
For speaking, press the button down slightly with your finger so as to enable phonation by means of the closure **●**.

CAUTION!
Do not use any fatty creams!
Otherwise, secure attachment is no longer ensured.

V. SERVICE LIFE
The maximum period of use / wearing period is 24 hours.
The tracheostoma protection can also be changed several times a day, as necessary.

VI. LEGAL NOTICES
The manufacturer Andreas FahI Medizintechnik-Vertrieb GmbH accepts no liability for damage caused by unauthorised product modifications, repairs not carried out by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products contrary to the provisions of these instructions for use.
Should a serious adverse event occur in connection with this product of Andreas FahI Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient are domiciled.
The manufacturer reserves the right to make changes to the product at any time without notice.
LARYNGOFIX® / TRACHEOSTOMA-FIX® is a trademark and brand of Andreas FahI Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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IT **I. FOREWORD**
These instructions for use are valid for LARYNGOFIX® / TRACHEOFIX / TRACHEOSTOMA-FIX® tracheostoma protection.
The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.
Please read the instructions for use carefully before using the product for the first time!
II. INTENDED USE
LARYNGOFIX® / TRACHEOFIX / TRACHEOSTOMA-FIX® filter the breathing air, thereby reducing the risk of particles entering the respiratory tract.
They store the moisture and warmth of the exhaled air in the filter medium and emit these to the inhaled air again during inspiration.
We supply LARYNGOFIX® / TRACHEOFIX / TRACHEOSTOMA-FIX® in various sizes.
With a button LARYNGOFIX® TOUCH enables closure of the tracheostoma, and hence phonation.

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