
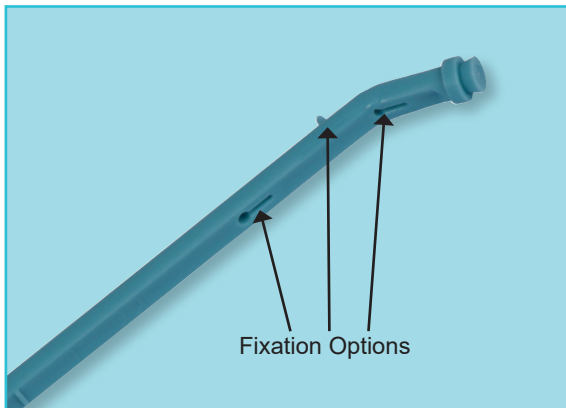



LARYVOX® INSERTER SPECIAL

Use with capsule technique

- ▶ LARYVOX® INSERTER SPECIAL is used for optimal antegrade placement or replacement of a shunt valve by application of the gel capsule technique in cases of
 - ▶ deep-positioned fistulas or fistulas with oblique course
 - ▶ constricted fistula channel
 - ▶ particularly small tracheostoma for optimising the viewing conditions
- ▶ The placement instrument has three different fixation options to allow easy and safe placement of various shunt valve types
- ▶ Effortless insertion of the shunt valve is enabled by application of the gel capsule technique.
- ▶ The shunt valve is introduced atraumatically into the shunt channel with the aid of the gel capsule. A drop of water-soluble local anaesthetic can be applied directly prior to placement in order to accelerate the dissolution of the gel capsule in the oesophagus.

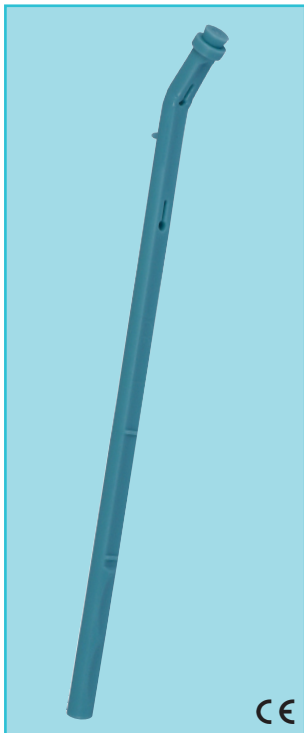

Placement Instrument

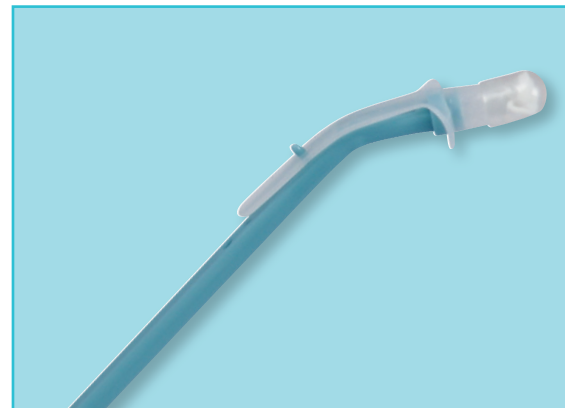

Placement Instrument incl. Shunt-Valve and Gel Caps

Order information:

LARYVOX® INSERTER SPECIAL

REF 25920